POWER X2

Powered Ambulance Cot

FERNO®

February 2026 | 234-3921-02

ENG FRA ITA SPA DEU

ALL	p. 1-17	
ENG	p.18-27	Instructions for use
FRA	p.28-37	Instructions d'Utilisation
ITA	p.38-47	Istruzioni per l'uso
ESP	p.48-57	Instrucciones de uso
DEU	p.58-68	Gebrauchsanweisung

Ferno.com
234-3968

Read and Retain for Future Reference

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Charge battery immediately upon receipt

Charger la batterie dès sa réception
Caricare immediatamente la batteria al ricevimento
Cargue la batería inmediatamente después de recibirla
Akku sofort nach Erhalt laden

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
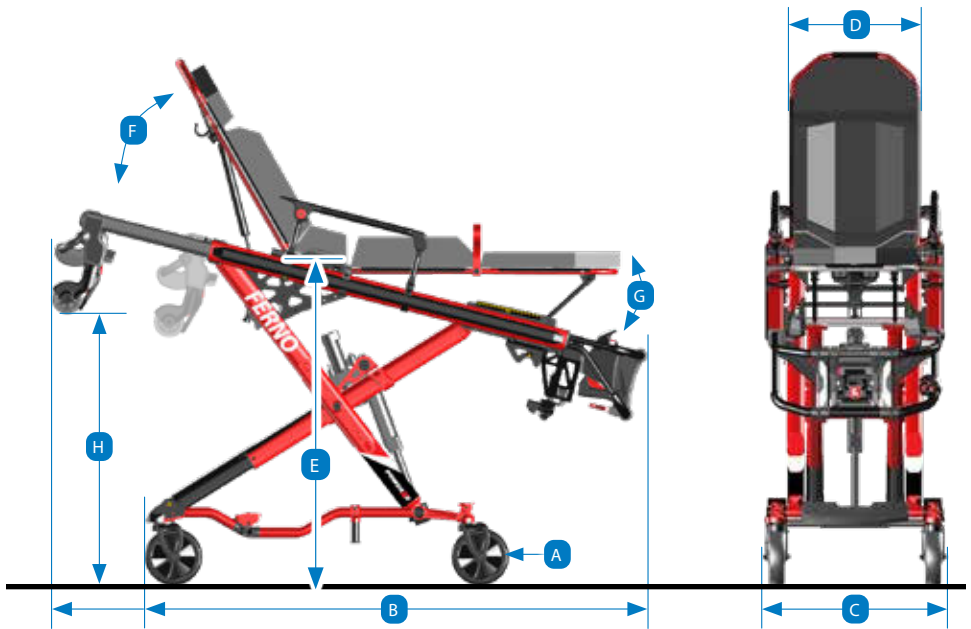
Ferno (UK) Ltd.
Ferno House, Stubs Beck Lane, Cleckheaton,
West Yorkshire, BD19 4TZ, UK
+44 (0) 1274 851 999






POWER X2	Metric	Imperial
A	193 x 51 mm	7.6" x 2"
B	1664 mm – 2030 mm	65.5" – 80"
C	610 mm	24"
D	435 mm	17"
E	402 mm – 1120 mm	16" – 44"
F		0°–77°
G		+15°; 30°
H	902 mm	35.5"
	318 kg/50 st	700 lb
	500 kg/78.6 st	1,100 lb
	66 kg	146 lb








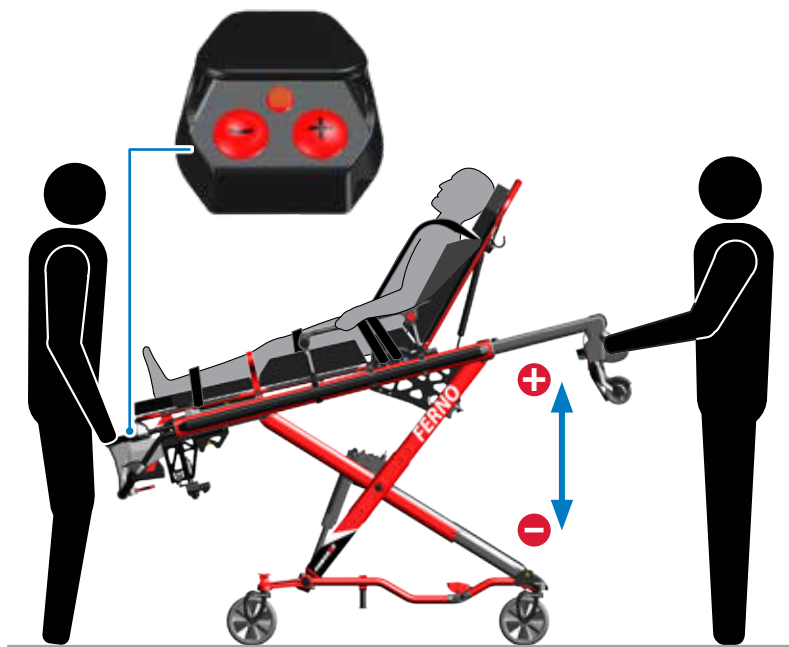
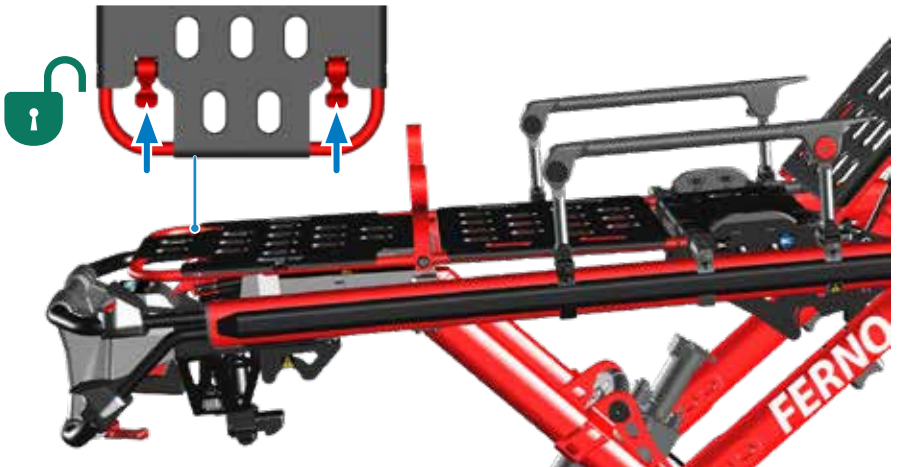
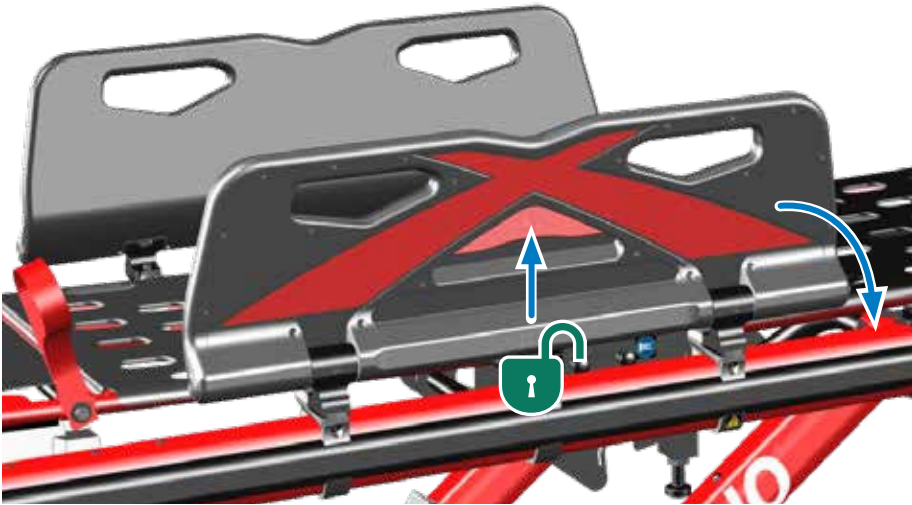
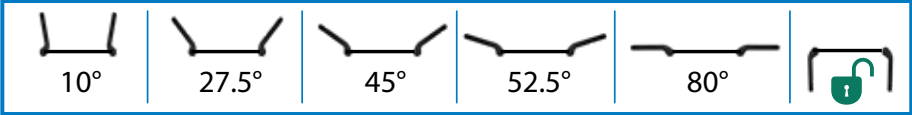
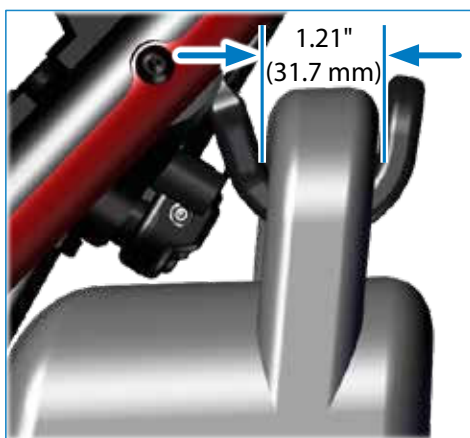
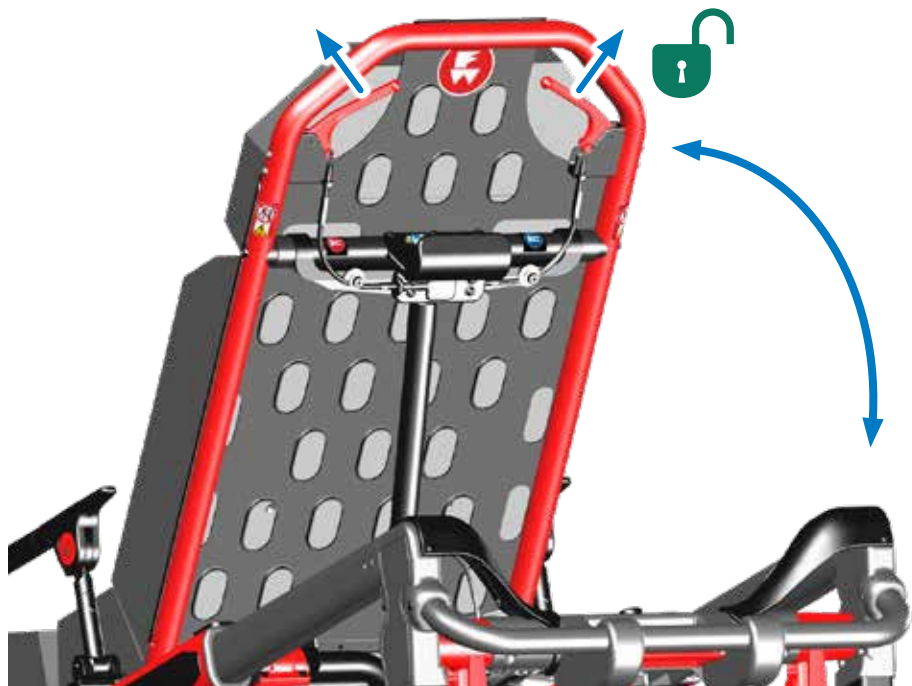
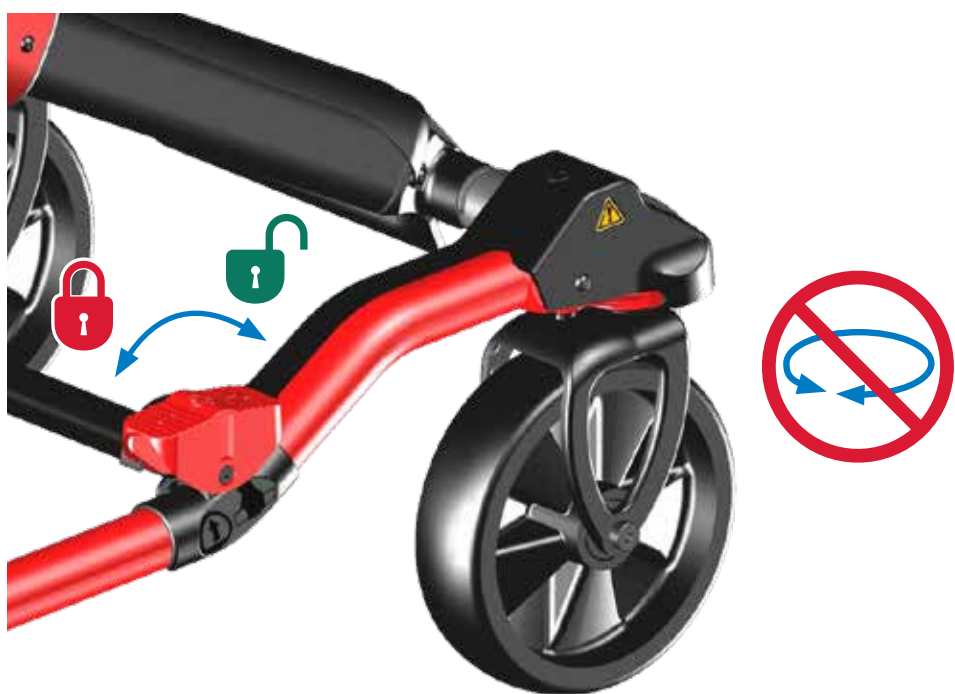
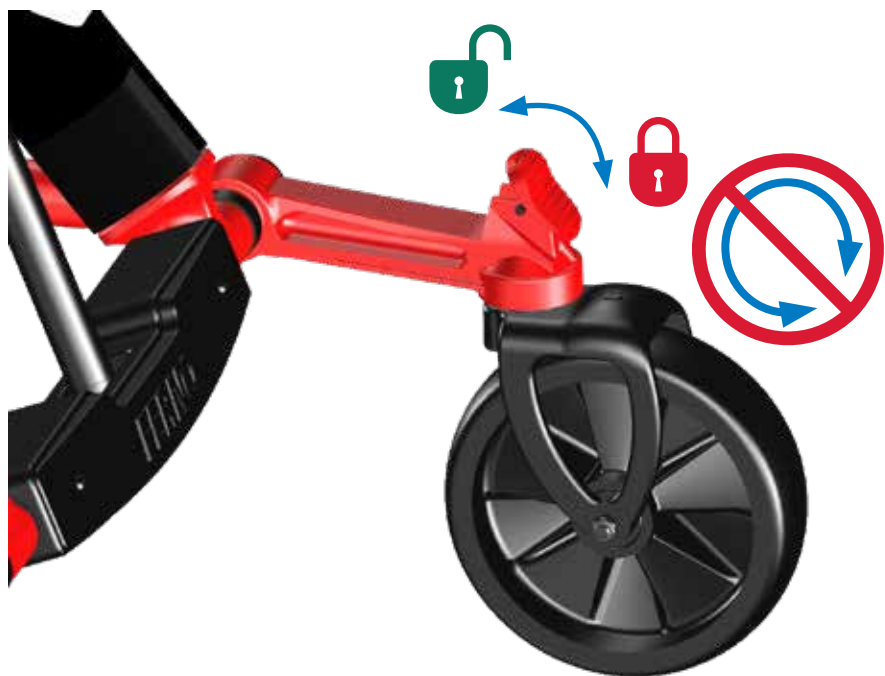
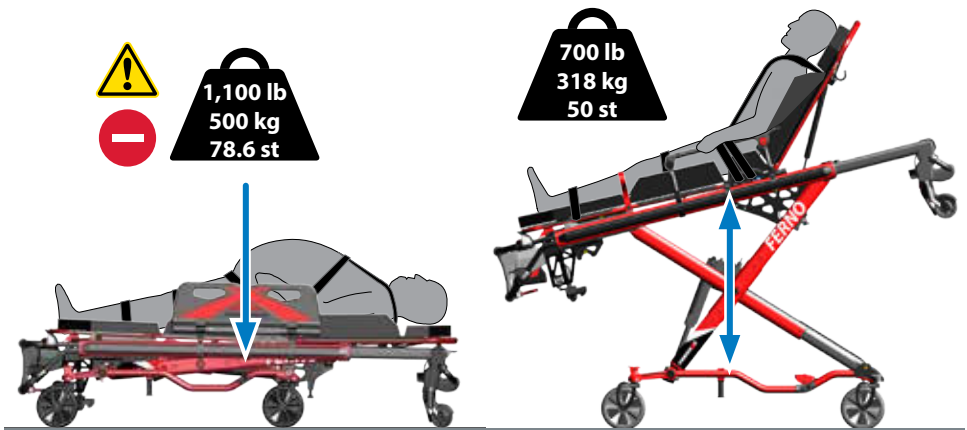
Manual Fastening System | Systèmes de fixation manuels | Sistemi di fissaggio manuali |
Manual Sistemas de Anclaje | Manuelle Befestigungssysteme

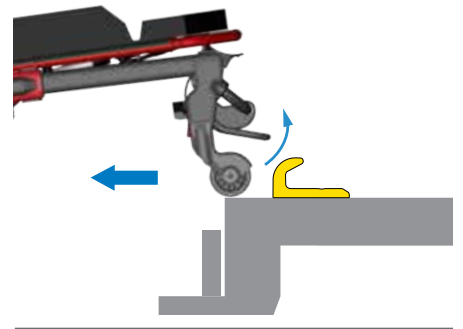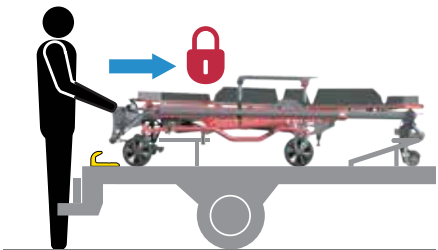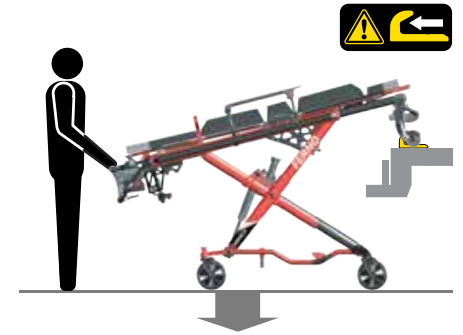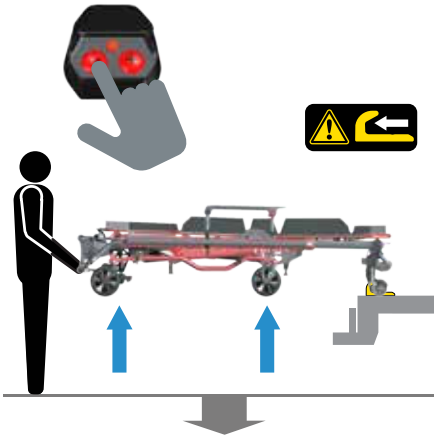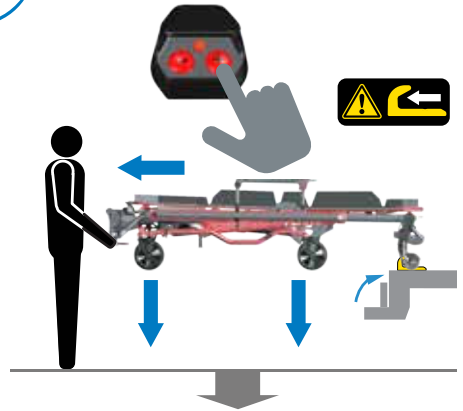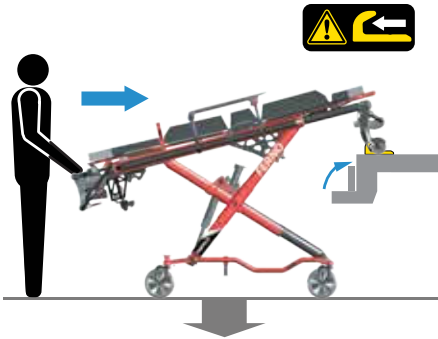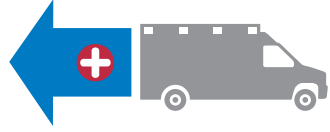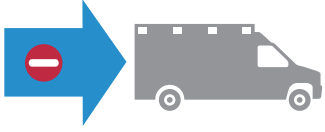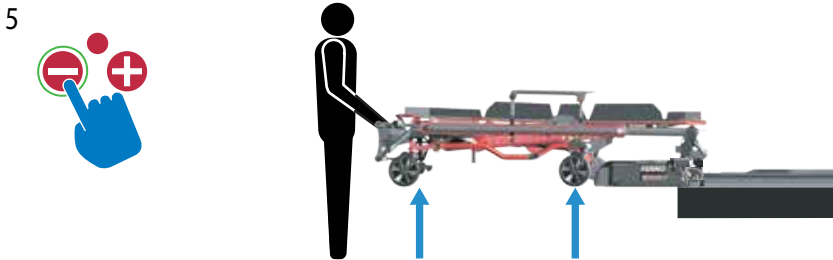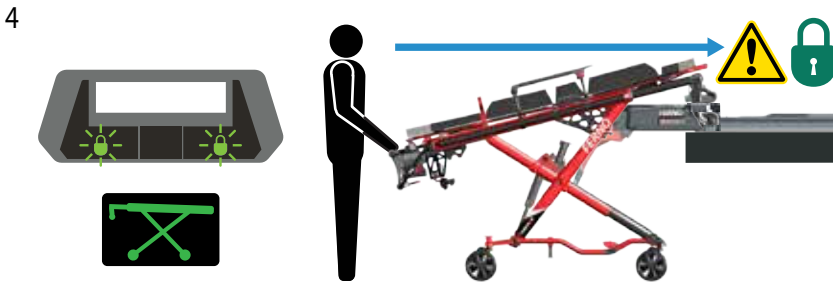
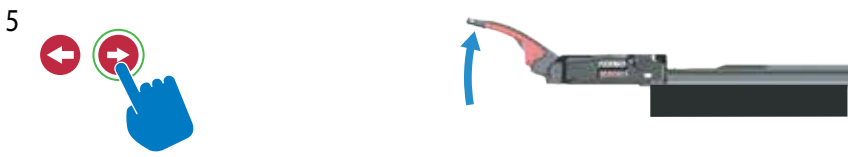
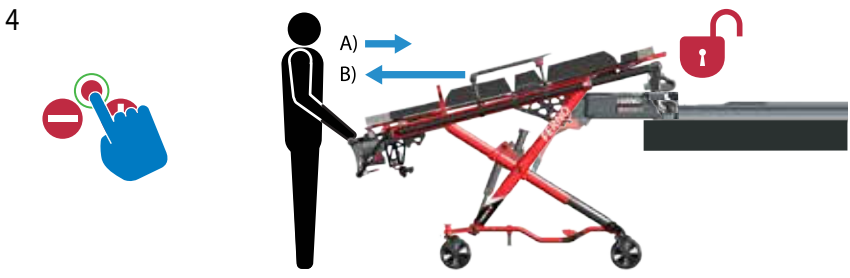
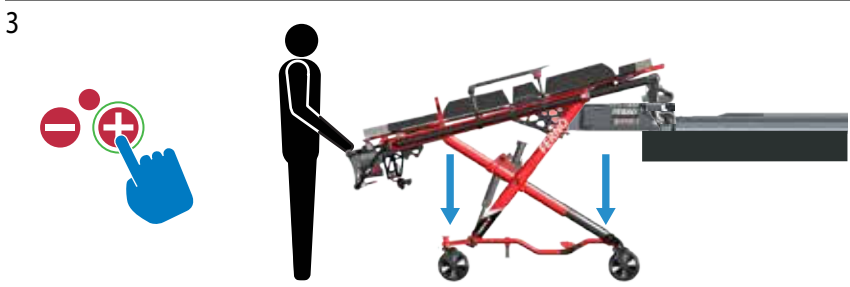
Power F2 Loading (Power Drive Disengaged) |

Chargement (Sans Système Power Drive) | Carico (Senza Power Drive) |





Cargando (Sin Power Drive) | Laden (Ohne Power Drive)

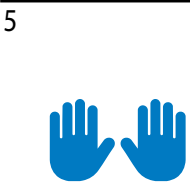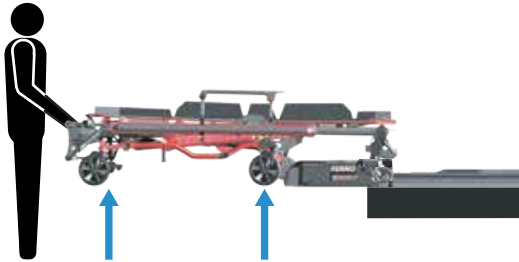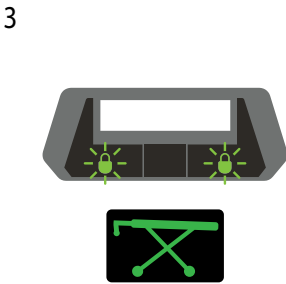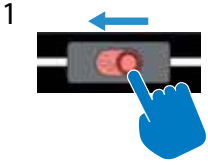Power F2 Unloading (Power Drive Disengaged) |

Déchargement (Sans Power Drive) | Scarico (Senza Power Drive) |

Descargando (Sin Power Drive) | Entladen (Ohne Power Drive)

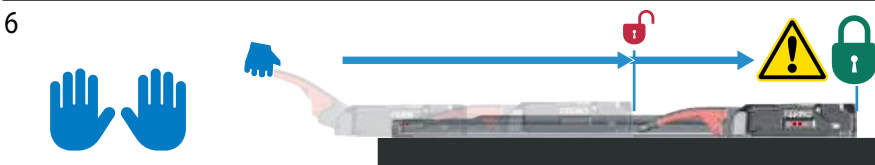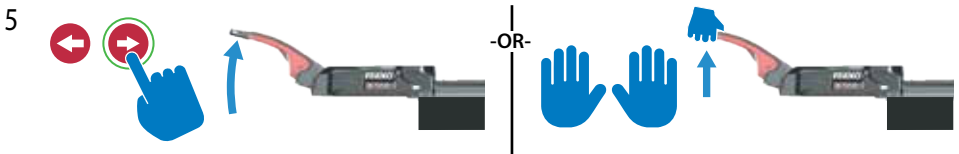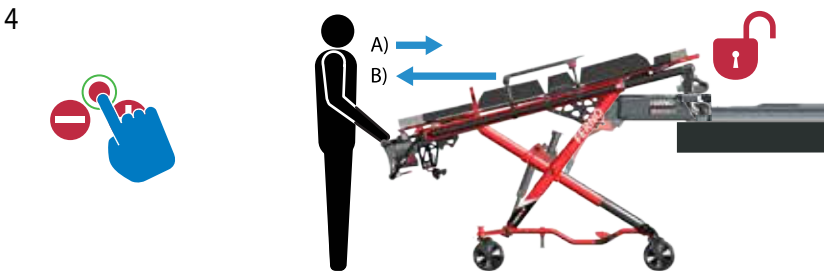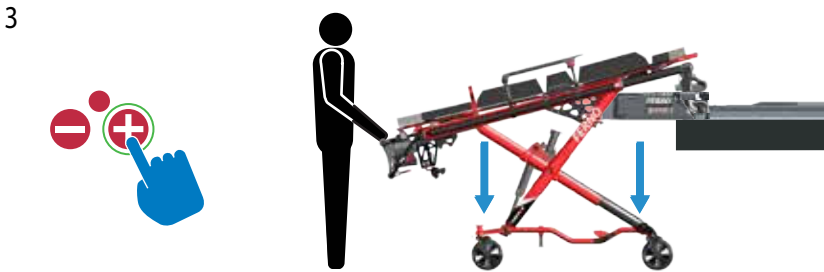Loading (Unpaired Cot, Power Drive Disengaged) |

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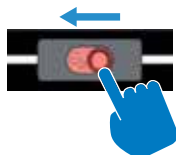
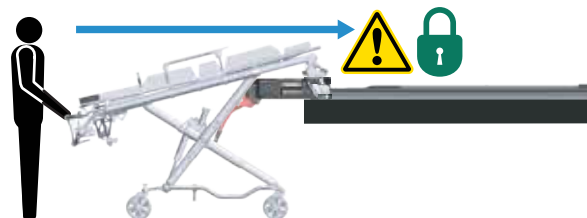
5

6





Unloading (Unpaired Cot, Power Drive Disengaged) |

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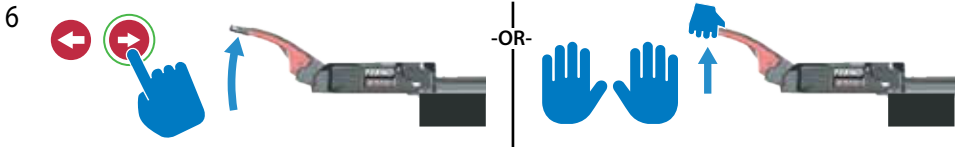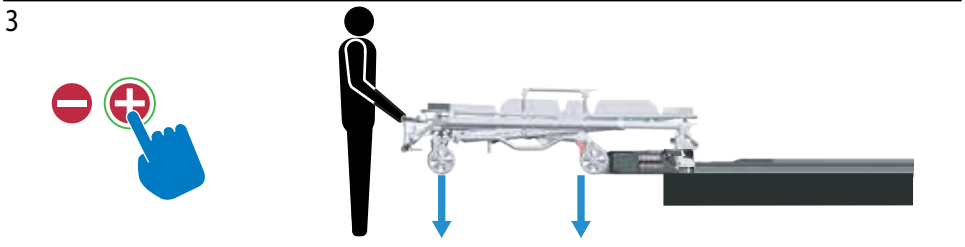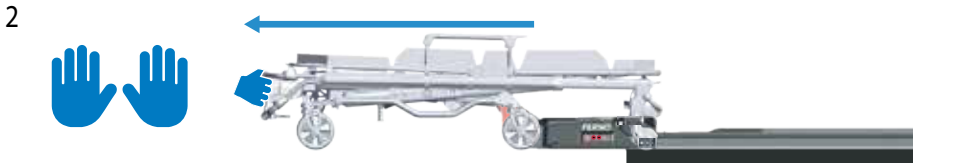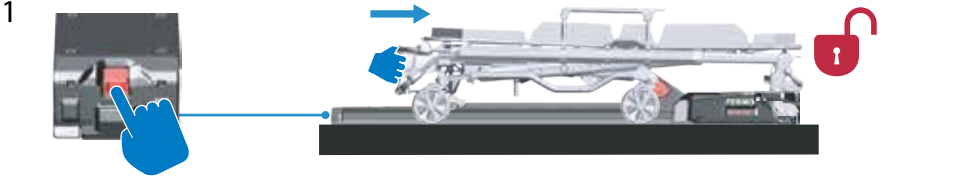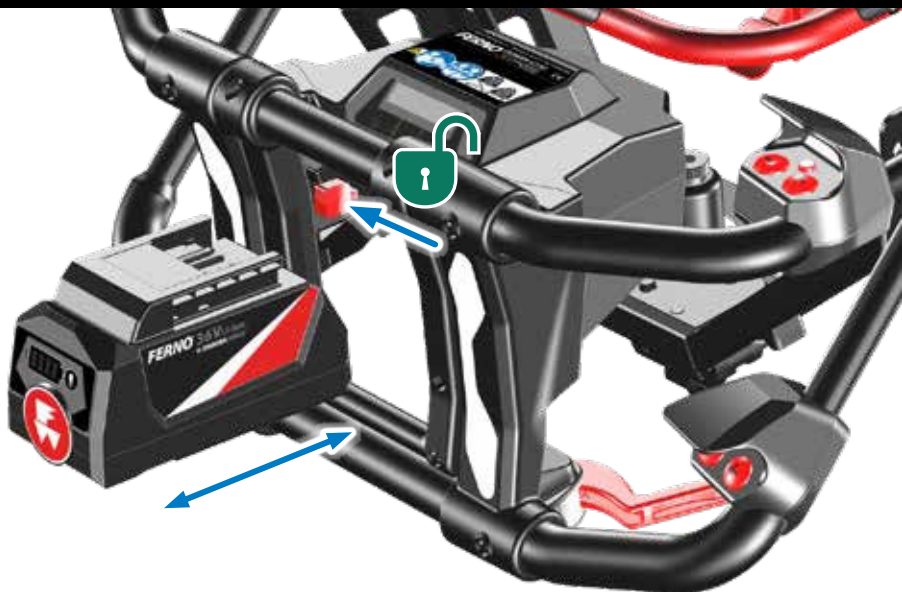
Press 1 second = Scene Lights Low

Press 1 second = Scene Lights OFF

Press both action buttons 10 seconds to set load height

Press and hold the button to unlock paired POWER X2 from Power F2

Display

CC Cycle Count

ERR Error

1 Max/Min position error

2 Cot not moving when motor on

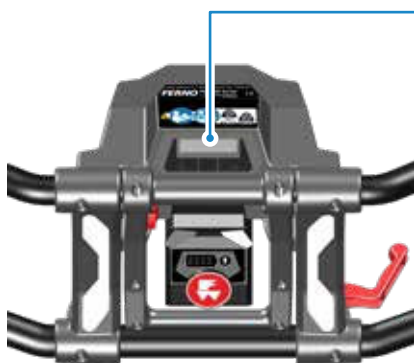
3 Motor over-current

4 Battery charging error

5 Stuck button error

6 Cot position error


7 Communications error

234-3968

SAFETY INFORMATION

Symbol	Definition	Symbol	Definition
	General warning of potential injury. Warning safety alerts indicate a potentially hazardous situation that, if not avoided, could result in injury or death.		Refer to instructions for use (ANSI Z535/ISO 3864)
	Pinch point; keep hands clear		Two trained operators required
	Do not open. No user serviceable parts.		Lubricate
	Do not push cot by the backrest		Pressure-wash permitted (cot only)
	Do not lubricate		Disinfecting
	Do not pressure wash (Battery)		Cleaning
	Remove items from equipment hook before transport		Lock
	Scene lights bright		Unlock
	Scene lights Low		Load Capacity/Safe Working Load
	Scene lights OFF		Load capacity in raised position
	Extend legs		Load capacity (fully lowered)
	Retract legs		Product weight
	Action button	IP56	Ingress Protection Rating (water and dust)
			
(Either lock) POWER X2 Paired with Ferno Connect		POWER X2 Not Paired with Ferno Connect	

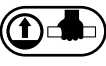Symbol	Definition	Symbol	Definition
	Consider instructions for use (ISO 7000)		Directional wheel lock
	Read Users' manual (ISO 7000)		Temperature limit (low/high)
	Follow training protocol		Check button (battery)
	Restraint attachment point		Service required (battery)
	Lift here (non-powered lifting)		
	European Directive 2012/19/EU on Waste Electrical and Electronic Equipment. Do not dispose of product in municipal waste. See "Disposal" on page 26.		

 **WARNING**

Professional use by trained operators only. Maintain control of product and controls. Direct helpers where to grasp to avoid pinch points. Consider terrain and use a lowered position to minimize potential to tip.

Exceeding the load capacity can cause injury and damage. Inspect the cot if the load capacity has been exceeded.

Use the cot only for its intended purpose. Improper use, operation, or maintenance can cause injury. Use, operate, and maintain only as instructed.

An unattended or unrestrained patient can be injured. Always stay with the patient and use patient restraints.

Rolling sideways or pushing by the backrest can cause the cot to tip. Roll only toward the control or operator end and grasp the main frame when rolling.

Sheets or other objects can interfere or become entangled in mechanisms and cause injury or malfunction. Tuck sheets between the mattress and the patient surface. Keep restraints and other articles above the mattress.

Failure to use the safety hook (provided) can cause injury. Install and use the safety hook as described in this manual. Operators must visually confirm the safety bar on the cot is captured by the safety hook during every loading and unloading sequence.

Improper parts, modifications or items not designed for the cot can cause injury. Use only Ferno-approved parts and service. Attach only approved items to the cot. Do not modify the cot.

Improper charging can cause injury and damage. Charge the battery only with an approved adapter and Ferno Battery Charger or compatible 36VDC Integrated Charging System (ICS).

Modifying the battery can cause injury and damage. Use the battery only as designed by Ferno.

Unsecured equipment can cause injury. Remove equipment from the equipment hook before transport inside a motor vehicle and secure in an appropriate device.

Improper bariatric transport can cause injury to the patient, operators and helpers. Follow local protocols and the instructions in this manual. Keep the cot folded when exceeding the standard load capacity. Use as many additional operators and helpers as needed to safely transport the patient.

OPERATOR TRAINING

Operator Training

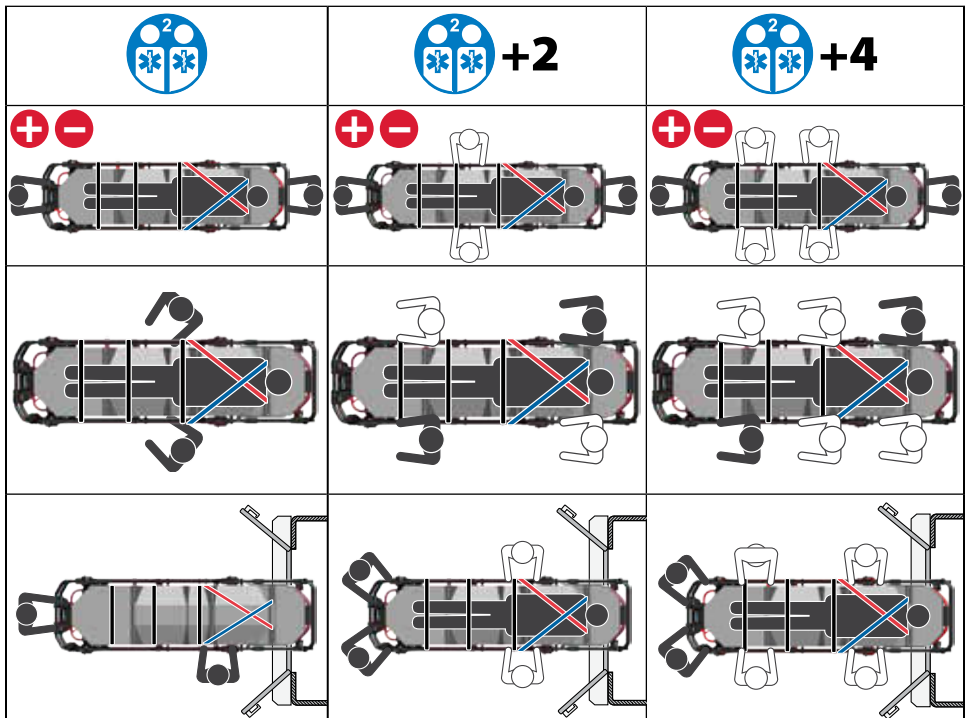
Operators must: read and understand this manual, have the physical ability to assist the patient, be trained on proper cot use, be trained on emergency-medical service and patient-handling procedures, and practice with the cot before using it with a patient. Keep training records.

Daily Operator Duties

Unload the cot from the ambulance and inspect it for readiness, rotate the cot in a circle to verify it rolls easily, and check the function of the manual back-up handle. Load the cot into ambulance and confirm it is secure in the fastening system. Confirm the main battery is charged.

Using Additional Help

Operation requires a minimum of two trained operators. Use help when working with heavy loads (patient plus equipment), or when operating the cot manually. Trained operators stand at the ends of the cot, maintain control of the cot, operate the controls, and direct helpers on where to stand and how to grasp the cot.

USING THE COT

Description

The Ferno® POWER X2 cot is an emergency patient-handling device designed to transport a patient in a ground-based ambulance. It is for professional use by a minimum of two trained operators. It is designed for roll-in loading to help reduce the risk of back injury to medical service personnel. The cot is for use with the Ferno PRO F1 or Model 175 (antler and rail) fastening systems (not included).

General Guidelines for Use

A minimum of two trained operators is required. Medical advice is beyond the parameters of this manual. It is the operators' responsibility to ensure safe practices for the patient and themselves.

- Operators must follow standard emergency patient-handling procedures when operating the cot, communicate with one another and use coordinated movements to operate the cot.
- Work together and maintain control of the cot at all times.
- Select and use appropriate, compatible patient restraints to secure the patient on the cot. A list of compatible Ferno patient restraints is below.
- Stay with the patient at all times. Communicate with the patient before making changes to the cot height, and during the loading and unloading procedures.
- Use additional trained helpers as needed. See "Operator Focus".
- The vehicle must have a compatible Ferno fastening system installed. Read the fastening system users' manual for instructions on using the fastening system.

Ferno Restraint	Patient Weight	
Kangoofix Neonate Transfer Aide	1.6-5 kg	3.5-11 lb
Model 676 MEDKids™ Baby Board	0.91-6.4 kg	2-15 lb
Neomate Infant Transfer Aide	2.3-6.4 kg	5-14 lb
Model 678 PediMate®	4.5-18.1 kg	10-40 lb
Model 677 MEDKids™ Pediatric Sleeve	5.4-27 kg	12-60 lb
PediMate+	4.5-45.4 kg	10-100 lb
X-Restraint system	Up to Rated Load Capacity	

MAINTENANCE

NOTICE

Lubricating parts that should not be lubricated allows dirt and foreign particles to collect on those parts, resulting in damage.

Do not use disinfectants or cleaners containing phenolics or iodines. If bleach is used, rinse with plenty of clear water to remove bleach residue.

Do not immerse restraint metal buckles, slide bars, and clips in liquids or by washing-machine action. See the restraint users' manuals.

Minimum Maintenance	Each Use or Daily	As Needed	Monthly
Daily Operator Duties	•		
Disinfecting	•		
Cleaning		•	
Pressure-Washing		•	
Inspecting		•	•

Maintenance Schedule

The cot requires regular maintenance. Set up and follow a maintenance schedule. Refer to table for minimum intervals for maintenance. Keep maintenance records. When using maintenance products, follow the manufacturers' directions and read the manufacturers' material safety data sheets.

Service Life

Factors affecting product service life include frequency of use, local environment (temperature, humidity, etc.); maintenance (adherence to the inspection and maintenance schedule); and other factors. Follow the disinfecting, cleaning, and inspection procedures in this manual to ensure long life for the cot.

Bloodborne Disease Notice

To reduce the risk of exposure to blood or other potentially infectious materials when using the Ferno product, follow the disinfecting and cleaning instructions in this manual.

Battery Care

The battery is water-resistant and is suitable for outdoor use. Do not submerge the battery in water. Remove the battery from the cot before pressure-washing the cot. Refer to the battery users' manual for disinfecting and cleaning instructions. Recycle the battery when it can no longer maintain a charge. Do not dispose of batteries with municipal waste. Replace a damaged or cracked battery.

Disinfecting and Cleaning

Remove the mattress and restraints as needed. Refer to the restraints users' manuals to disinfect and clean restraints. Reattach mattress and restraints only when clean and dry. Inspect all items for damage while disinfecting and cleaning.

Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time.

Hand clean with warm water and a mild detergent. Rinse with warm, clear water. Dry with a towel or allow it to air-dry.

Pressure-Washing

The cot is water-resistant (IP56). When pressure-washing, do not remove the battery or spray water into cables or forcibly at battery housing.

1,200 psi
457 mm 18"

Use lower pressure,
increase distance

Inspecting

Have your service's equipment maintenance personnel inspect the cot regularly following the checklist. If inspection shows damage or excessive wear, remove the cot from service until repair is made.

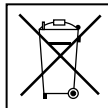
- Are all components present?
- Is cot free of excessive wear?
- Are all fasteners (screws, nuts, bolts) secure?
- Do all moving parts operate smoothly and properly?
- Does cot load and unload properly?
- Does manual mode handle function properly?
- Does the safety hook engage the safety bar?
- Does cot roll smoothly?
- Do wheels have some tread?
- Do wheel locks engage properly and stop wheel rotation or swiveling?
- Are patient restraints properly installed?
- Is patient-restraint webbing in good condition with no cuts or frayed edges?
- Are patient-restraint buckles free of visible damage and do they operate properly?
- Does ambulance have an approved Ferno fastening system and safety hook?
- Do installed accessories operate properly without interfering with cot operation?

Adverse Event Notice

In the event of an adverse event or serious incident related to the use of this device, the end user/operator must report the incident to Ferno at 1-877-733-0911, or via email at tscoordinator@ferno.com. If the incident occurred outside of the U.S., report it to Ferno's Authorized Representative for that area.

Disposal

Prior to disposal, clean and disinfect the product according to the instructions in this manual. Recycle or dispose of the product following local protocols and regulations. Do not dispose of the product in municipal waste. Remove and recycle batteries and electronic components per local regulations.

Unique Device Identification (UDI)

Disclaimer

This manual contains general instructions for the use, operation and care of this product. Instructions are not all-inclusive. Safe and proper use is solely at the discretion of the user. Safety information is included as a service to the user. All other safety measures taken by the user should be within and under consideration of applicable regulations and local protocol. Training on the proper use of this product must be provided before using this product in an actual situation.

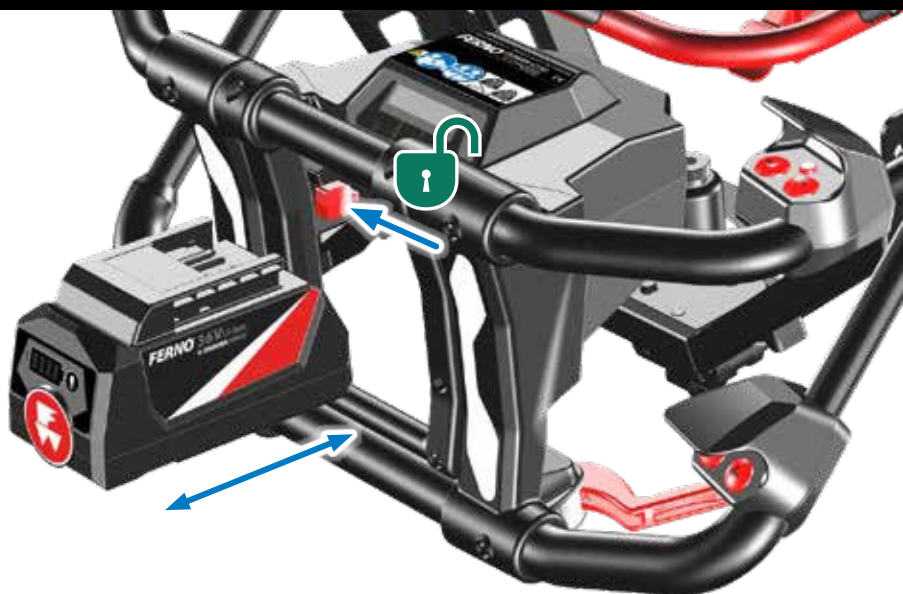
Retain this manual for future reference. Include it with the product in the event of transfer to new users. Additional free copies are available upon request from Ferno.

Proprietary Notice

The information disclosed in this manual is the property of Ferno. Ferno reserves all intellectual property rights, proprietary design rights, manufacturing rights, reproduction use rights, and sales use rights thereto, and to any article disclosed therein except to the extent those rights are expressly granted to others or where not applicable to vendor proprietary parts.

Limited Warranty Statement

Ferno products are covered by a limited warranty, printed on the invoice. Complete terms and conditions, and limitations of liability, are available upon request from Ferno.

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