



Technimount System

445, Papin St. – Office 103
Quebec City (Quebec), Canada
G1P 3T8
Telephone: +1-581-998-9820
Fax: 1-855-339-6351

info@technimount.com

www.technimount.com

INSTRUCTION GUIDE

FOR

BRACKET PRO SERIE 20™

MOUNTING SYSTEM FOR MEDICAL DEVICE

INSTALLATION FOR: **ZOLL E Series™**

Version 3.0 English

INSTRUCTIONS FOR BRACKET PRO SERIE 20™

WARNING:

We do our best to manufacture a product with high quality. However, it may happen that some section on the parts can be very cutting and sharp, please handle with care.

Description:



This bracket mounting system comes in two sections; one section (part 1) to be installed on the medical device (*ZOLL, E Series™*); and the second section (part 2) to be installed into the vehicle or on crash cart. This instruction concerns the installation on the bracket mounting on the device only. **The second part, the support base mounting (part 2) must be installed by a professional, on certify anchoring points on a horizontal surface.** To fix it, 4 points of insertion of bolt must be used; we recommend using those four points.

Contents:

- 1 X Bracket mounting round swivel (*to be installed under the medical device*) (figure 1)
- 1 X Support mounting (base plate) to be installed on a surface in the vehicle or crash cart (figure 2)
- 4 X Screw Flat Head 10-32 x 1"
- 2 X "Hat" washer or spacer (front)
- 2 X "Puck" washer or spacer (rear)
- 1 X Instruction sheet

**** Screws for support base mounting are NOT provided with your package; installer will choose the appropriate screw, or bolt and nut according to the anchoring points. ****

Tools required:

- 1 x Philips screw driver (star) ; with diameter of the shaft smaller than 5 mm or 3/16

Identification of the product



Figure 1 – **Bracket** mounting



Figure 2 – **Support** base mounting



Figure 3 – **Screws & insert**

Instructions for part one (bracket mounting on the medical device)



Step 1. (Removing the two round rubber protectors under the unit, covering the screws)

- ✓ Remove the round rubber protector with a screw driver or another specific tool.




- ✓ Put the rubber protector away, it won't be usable for this assembly.

Step 2. (Remove the four screws use for attaching front carry case)

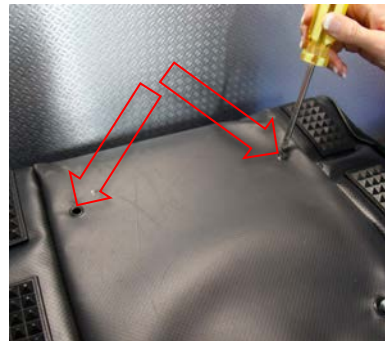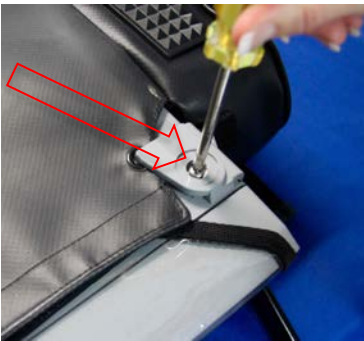

- ✓ Put the four screws away, it won't be usable for this assembly. New ones will be provided.

Step 3. (Install the two front "hat" washers on the front of the device)

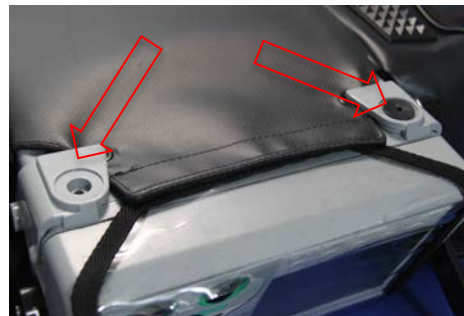

- ✓ Put the two front washers (spacer) in the hold, hat shape down into the hold.

Step 4. (Install the two rear “puck” washers on the front of the device)

- ✓ Put the spacer on the screw hold, and aligned the edge into the hold

Step 5. (Install the device bracket mounting) –

- ✓ Place the bracket under the unit, the “V” shape facing front the unit.
- ✓ Align and screw the 4 Philips screws 10-32 x 1” in the screw holds on the device.
- ✓ Review all the screws installed and tighten normally in an alternative sequence.



Here is the bottom view of the device with bracket installed and the handle can be fully deployed and functional

Instructions for part two (Support mounting on a surface in the vehicle)

WARNING:

The second part, the support base mounting (part 2) must be installed by a professional recognized in the industry, on certified mounting points on a horizontal surface. In order to fix it, 4 points of insertion of bolt must be used; we recommend using those four points.

NOTE:

PLEASE VERIFY YOUR LOCKING MECHANISM ON THE BASE PART PRIOR THE FINAL INSTALLATION ON YOUR SURFACE, IN ORDER TO DETECT ANY MALFUNCTION.



Final installation suggested

MAINTENANCE:

We recommend cleaning the disc under the medical device after each use, and the surface base mount as well, after each use. In order to prevent any malfunction or issue, clean carefully the components, at least once a month with a soft soap and linen, to remove any dust or small particules on the mounting bracket. **Do NOT disassembly the mechanism**, if you suspect any malfunction, please contact us.

If any question, please call your distributor or Technimount System at +1.581.998.9820 or email us at: techsupport@technimount.com

www.technimount.com

Thank you!