



**TECHNIMOUNT**  
EMS™

# BRACKET PRO SERIE® 280

## USER MANUAL

SAFETY AND FLEXIBILITY  
WHERE IT MATTERS MOST



## Copyrights

Copyright 2023 Technimount E.M.S. Holding Inc. (Technimount EMS) is an affiliated company of Technologies CGC Inc. All Rights Reserved. No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system, translated in any language, modified in any form or by any means, without the written permission of Technimount or its affiliate companies.

## Trademarks

- Safety Arm System™ and Technimount EMS™ are trademarks of Technologies CGC Inc.
- Bracket Pro Serie® and Safety MD-Transporter® are registered trademarks of Technologies CGC Inc.
- VOCSN® is a registered trademark of Ventec Life Systems.

Product and medical equipment manufacturer names are Trademarks™ or Registered Trademarks® of their respective holders. Technimount does not have a commercial relationship with these medical equipment manufacturers.

For any issues with your Technimount product, its components, or for any technical questions during the installation, operation, or maintenance, please contact Technical Support at [techsupport@technimount.com](mailto:techsupport@technimount.com).

## Contact Information

### Technimount E.M.S. Holding Inc.

C/O: Regulatory Affairs

3770 Jean Marchand Street, Suite 100-C

Quebec (QC) G2C 1Y6

Canada

[customerservice@technimount.com](mailto:customerservice@technimount.com)

[techsupport@technimount.com](mailto:techsupport@technimount.com)

[legal@technimount.com](mailto:legal@technimount.com)

[www.technimount.com](http://www.technimount.com)

T + 1 581.700.6613

TF + 1 888.639.2758

## Table of Contents

<b>Copyrights .....</b>	<b>ii</b>
<b>Trademarks .....</b>	<b>ii</b>
<b>Contact Information .....</b>	<b>ii</b>
<b>1. General Mentions and Considerations .....</b>	<b>4</b>
..... 1.1. Intended Use .....	4
..... 1.2. User Competency .....	4
..... 1.3. Warranties .....	4
..... 1.3.1. Warranty Policy .....	4
..... 1.3.2. Limited Warranty .....	5
..... 1.3.3. International Warranty Clause .....	5
..... 1.3.4. User Liability .....	5
..... 1.4. Claims .....	5
..... 1.4.1. Damaged or Defective Merchandise .....	5
..... 1.4.2. Return Policy .....	6
..... 1.4.3. Return of Material Authorization (RMA) .....	6
..... 1.4.4. Claim Process .....	6
<b>2. General Safety Guidelines .....</b>	<b>8</b>
..... 2.1. Symbols and Definitions .....	8
..... 2.2. Labels .....	9
..... 2.3. Safety Measures .....	9
<b>3. Technical Specifications .....</b>	<b>11</b>
<b>4. Unpack the Mounting System .....</b>	<b>12</b>
<b>5. Bracket Pro Serie 280 Illustrated Parts .....</b>	<b>13</b>
<b>6. Maintenance .....</b>	<b>14</b>
..... 6.1. Required Tools .....	15
..... 6.2. Condition-Based Maintenance Plan .....	15
<b>7. Spare Parts &amp; Upgrade Kits .....</b>	<b>17</b>

## 1. General Mentions and Considerations

Intended for the purchaser / supervisor, this user manual contains detailed product information and was designed to assist with the unpacking, assembly (when indicated) and maintenance of the Bracket Pro Serie 280 mounting system. It's content should be relayed to EMS personnel during training. This document should be used with the operating guide, which contains specific user related information such as the operating procedures and the daily safety checks.

**NOTE:** Technimount continually seeks advancements in product design and quality. While this user manual contains the most updated product information available at the time of printing, it may contain minor differences with the current version. For more information, please contact Customer Service at [customerservice@technimount.com](mailto:customerservice@technimount.com).

### 1.1. Intended Use

The Bracket Pro Serie 280 is designed to secure the Ventec VOCSN Multi-Function ventilator during critical care transport.

### 1.2. User Competency

The Bracket Pro Serie 280 is intended for EMS personnel who have received the proper training, necessary to operate the equipment in the field, according to its intended use, as per outlined in this user manual and operating guide. Please read all provided documentation thoroughly before using this equipment to ensure the safe operation of this equipment and provide a safe environment for patients and EMS personnel.

### 1.3. Warranties

#### 1.3.1. Warranty Policy

This statement constitutes Technimount's entire warranty policy with regards to the Technimount products. Technimount makes no other warranty or representation, neither expressed nor implied, except as stated herein. There is no warranty of merchantability or warranties of fitness for any particular purpose. Under no circumstances will Technimount be held liable hereunder for incidental or consequential damages, arising from or in any manner, related to sales or use of any such equipment.

Technimount E.M.S. Holding Inc. (Technimount) guarantees to the original "Purchaser" of the "Product" with which this "Limited warranty" is included, that the product will be free from "Defects" in workmanship and materials under normal use for a "Warranty period" of one (1) year from the product purchase date by the purchaser. During the warranty period, the product will be repaired or replaced according to the "Limited warranty" without charge to the purchaser for parts or labor. The parts and product may be repaired or replaced with new or refurbished parts or products. Herein this Limited Warranty, "Refurbished" means parts and products which have been returned to the factory, specifically. If the product is repaired or replaced within the warranty period, the greater of the remaining warranty period will apply, or three (3) months from the date of repair or replacement. If the product is repaired or replaced after the warranty period has expired, the warranty period for the repair or replacement will expire three (3) months after the repair or replacement date.

### **1.3.2. Limited Warranty**

Technimount products are intended to retain medical equipment in place in the case of a single crash impact. Technimount products must not be reused if involved in a crash and must thereafter be replaced. If the end user uses a Technimount product following a crash, it is at the end user's own risk. Technimount will not be held liable.

The limited warranty does not apply to normal wear that could result from normal use. It does not apply when the product or any of its components has been disassembled or repaired by someone not authorized by "Technimount" or if the seal has been broken. It does not cover repair or the replacement of any product or part thereof damaged by neglect, misuse, moisture, liquids, exposure to heat, accidents, abuse, and non-compliance with the instructions for installation and use provided with the product.

The limited warranty does not cover physical damage to the surface of the product. The decision to repair, replace or refuse the coverage is final and at the sole discretion of Technimount, without any compensation or obligation from Technimount. The product defined as a "Mounting system" or "Bracket system" used for clipping and attaching medical equipment, is specifically designed to fill this requirement. Any other use will void the warranty, and Technimount shall not be held liable on any claim if the product has been modified or adapted for use.

### **1.3.3. International Warranty Clause**

This warranty abides by the Canadian domestic policy. Warranty outside Canada may vary by country. Please contact Customer Service at [customerservice@technimount.com](mailto:customerservice@technimount.com) for more information.

### **1.3.4. User Liability**

The purchaser and administrator are responsible to validate regulations and standards for safety in their region, to comply with applicable safety regulations. Technimount is not responsible to inform the purchaser or the administrator of any applicable legislation for safety in their area.

The purchaser and administrator are responsible for providing proper training on the installation, operation, and maintenance of Technimount products to anyone susceptible of contact with the product prior to use.

## **1.4. Claims**

### **1.4.1. Damaged or Defective Merchandise**

ICC Regulations require that claims for damaged merchandise must be made with the carrier within fifteen (15) days of the reception date. Do not accept damaged merchandise unless such damage is noted on the delivery receipt at the time of reception. Upon prompt notification, Technimount will file a freight claim with the appropriate carrier for damages incurred. Claims are limited in amount to the actual replacement cost. If the claim has not been received by Technimount within the fifteen (15) day period following the date of delivery, or the damage was not noted on the delivery receipt at the time of receipt, the customer will be responsible for the full payment of the original invoice.

Claims for any short or broken merchandise must be made within thirty (30) days of invoicing. Refer to section 1.4.4 for the detailed claim process.

#### 1.4.2. Return Policy

Technimount's mounting and bracket systems for portable medical equipment may be returned up to sixty (60) days from the reception date, if:

- The received product does not match what was originally ordered.
- The product does not meet the Technimount technical sheet specifications.
- The product is not compatible with the equipment on which it was intended to be installed onto.

To return a Technimount product,

- A Return of Merchandise Authorized (RMA) must be requested and approved by Technimount prior to returning the product.
- Products must be returned undamaged and in its original packaging, appropriately identified with the approved RMA number. Returns will not be approved on a modified or damaged item.
- Charges may apply if the package received is damaged or items are missing.
- Purchaser is responsible for a restocking fee (refer to Table 1).

**Table 1:** Restocking fees

RESTOCKING FEES	
Prior to thirty (30) days	10%
Prior to forty-five (45) days	25%
Prior to sixty (60) days	30%

For any manufacturing defect, refer to the conditions within the warranty policy in section 1.3.1 or contact Customer Service at [customerservice@technimount.com](mailto:customerservice@technimount.com) for additional information.

#### 1.4.3. Return of Material Authorization (RMA)

The Technimount Customer Service department is responsible for all merchandise returns and will provide a Return of Merchandise Authorization (RMA) number, upon approval. The RMA must be printed and placed on the returned merchandise. Technimount reserves the right to charge shipping and restocking fees (refer to Table 1) for the returned items. Special, modified, or discontinued items are not subject to returns.

#### 1.4.4. Claim Process

Upon reception of the returned merchandise, a thorough inspection will be performed. If the merchandise is compliant with the return policy or it is found that the product is defective, Technimount will take corrective actions and close the claim. If, however, it is found that the product is not defective, but rather misused or abused, the product will not be covered by the warranty. Details of our findings and conclusions will be provided shortly thereafter. To submit a claim, contact Customer Service at [customerservice@technimount.com](mailto:customerservice@technimount.com) to obtain a Return of Material Authorization (RMA) form and return instructions.

## 2. General Safety Guidelines

Pictograms, safety symbols and labels are used to alert the user to a potentially hazardous situation, which, if not avoided, may result in minor or moderate injury to the user or patient or cause damage to the equipment or other property. This includes the special care necessary for the safe and effective use of the device to avoid damage that may occur from use or misuse. The terms "Warning" and "Caution" herein carry special meaning and should be carefully reviewed.

**WARNING** - Indicates a hazardous situation that, if not avoided, could result in death or serious injury.

**CAUTION** - Indicates a hazardous situation that, if not avoided, could result in minor or moderate injury.

Always read and abide by all the safety guidelines identified within this document.

### 2.1. Symbols and Definitions



**WARNING - General warning**

Alerts the reader of a potentially hazardous situation, which, if not avoided, may result in death or serious injury to the user or patient or cause damage to the equipment or other property. This includes the special care necessary for the safe and effective use of the equipment to avoid damage that may occur from use or misuse.



**WARNING - Safe working load / Load balance**

Indicates the total maximum charge for a safe use of equipment.



**CAUTION - General mandatory action**

Call for action. Alerts the reader to potential risk to people not following the mandatory action specified by the supplementary sign.




**CAUTION - Follow instructions for use**

Call for action. Reminds the reader to consult the user manual for information.

## 2.2. Labels

Labelling (if applicable) on the surface of the Technimount product, quickly identify potential risks, and provides information to the user. A serial number (Figure 1) can be seen on the Technimount product.

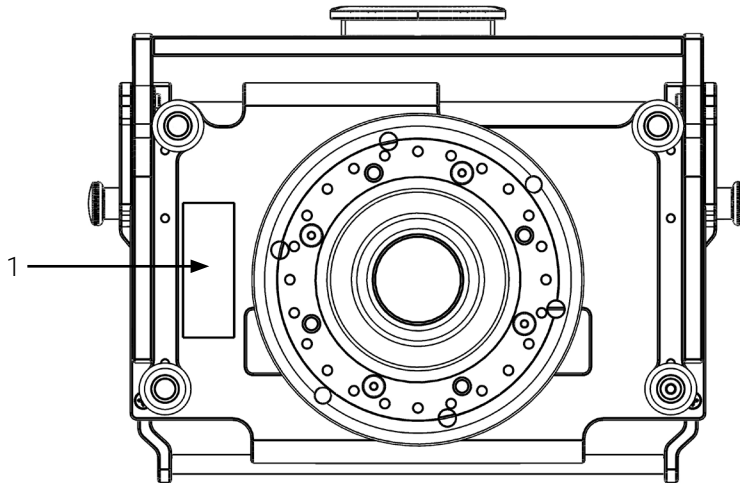

1. Serial number

**Figure 1:** Location of the serial number (bottom of bracket shown)



## 2.3. Safety Measures

Carefully read all safety measures herein before installing, operating or performing the maintenance of the Technimount product.



### **WARNING - Risk of injury**

Regulations and standards for safety are the sole responsibility of the end user. Ensure that the installation specifications meet the local and regional compliance requirements before use. Product incompatibility could cause unpredictable functioning resulting in injury to the patients or EMS personnel.



### **CAUTION - Safe handling and operation**

- The end user and administrator are responsible for providing proper training to any personnel susceptible of contact with the product prior to use. Installation, operation and maintenance should only be performed by trained personnel.
- Improper use of the Technimount product may damage the equipment or cause injury to the patients or EMS personnel.
- **Do not** modify the product, or any of its components and use only as per prescribed. Modifying the product or improper use could cause unpredictable functioning, resulting in injury to the patients or EMS personnel.



### **CAUTION - Safe practice**

- Always pay close attention to the condition of the safety mechanisms and fixation points, to prevent undue risk to the equipment, patients and EMS personnel. Follow the recommended maintenance plan and its guidelines, as prescribed in the user manual and operating guide.
- Always ensure that the medical equipment is secured in the bracket and that the locking mechanism is functional. before use.
- Always keep the user manual and operating guide within reach of the product, even if the equipment is subsequently sold, to prevent undue risk to the equipment, patients and EMS personnel. The user documentation is an integral part of the equipment.
- Practice installing, removing, and safely operating the medical equipment until the manipulations have been perfected, before use with patients. Improper use of any Technimount product may damage the equipment or cause injury to the patients or EMS personnel.



### **CAUTION - Working load/Load balance**

The Safe Working Load (SWL) is 18 lbs (8.2 kg). Do not overload the system.



### **CAUTION - Follow instructions for use**

- Always read and abide by all the safety guidelines identified, as well as follow instructions provided within this document.
- Refer to the medical device's user manual for safety precautions and user instructions for the safe use of the Ventec VOC SN ventilator.

### 3. Technical Specifications

<b>Product Name</b>	Bracket Pro Serie® 280
<b>Description</b>	Mounting solution designed to secure the Ventec VOCSN Multi-Function ventilator during critical care transport
<b>Part Number</b>	2800-10-VOCSN
<b>Features</b>	<ul style="list-style-type: none"> <li>- Multiple mounting configurations on surfaces, stretchers, and carts</li> <li>- Quick release knobs for easy installation and removal of the VOCSN ventilator</li> <li>- Robust bracket for EMS / CCT transport</li> <li>- Ground compatibility</li> </ul>
<b>Operating Environment</b>	EMS / CCT, Hospital / Clinic, Military
<b>Compliance</b>	SAE J3043 and AMD 028
<b>Compatible Cot / Stretcher<sup>1</sup></b>	Contact Customer Service at <a href="mailto:customerservice@technimount.com">customerservice@technimount.com</a> for compatible stretcher mounting solutions
<b>Compatible Mounting System</b>	Technimount Bracket Pro Serie® mounting systems with an anti-rotation micro disc and standard bottom disc
<b>Dimensions (W X D X H)</b>	12.7 in. x 9.3 in. x 11.05 in. (32.26 cm x 23.62 cm x 28.07 cm)
<b>Weight</b>	6.4 lbs (2.9 kg)
<b>Composition</b>	<ul style="list-style-type: none"> <li>- Bracket Pro Serie® 280: aluminum 5052-H32 and 6061-T6, silver anodized finish</li> <li>- Feet, bumpers: Acetal</li> </ul>
<b>Safe Working Load (SWL)</b>	18 lbs (8.2 kg)
<b>Operating Temperature</b>	- 31° F to 113° F (- 35° C to 45° C)
<b>Installation</b>	<ul style="list-style-type: none"> <li>- Multi-functional ventilator retained in mounting system by locking pull-down bar</li> <li>- Quick release knobs are used to easily release and remove the ventilator from the mounting system</li> </ul>
<b>Model &amp; Configuration Options</b>	Contact Customer Service at <a href="mailto:customerservice@technimount.com">customerservice@technimount.com</a>

<sup>1</sup> Product and medical equipment manufacturer names are Trademarks™ or Registered Trademarks® of their respective holders. Technimount does not have a commercial relationship with these manufacturers.


#### 4. Unpack the Mounting System

1. Inspect the shipping box(s) for signs of damages, before accepting shipment. Take pictures and report them promptly if applicable.
2. Move the shipping box(s) to the location of the installation.
3. Open the shipping box(s).
4. Unpack the box(s) and ensure that all shipping and packaging materials have been properly removed, prior to installation.

**NOTE:** Keep all packaging material for future use.

5. Identify all the components and hardware included for installation, then set aside. Refer to Section 5 for the illustrated parts, if needed.
6. Inspect the items for signs of damages. Take pictures and report them promptly if applicable.

## 5. Bracket Pro Serie 280 Illustrated Parts

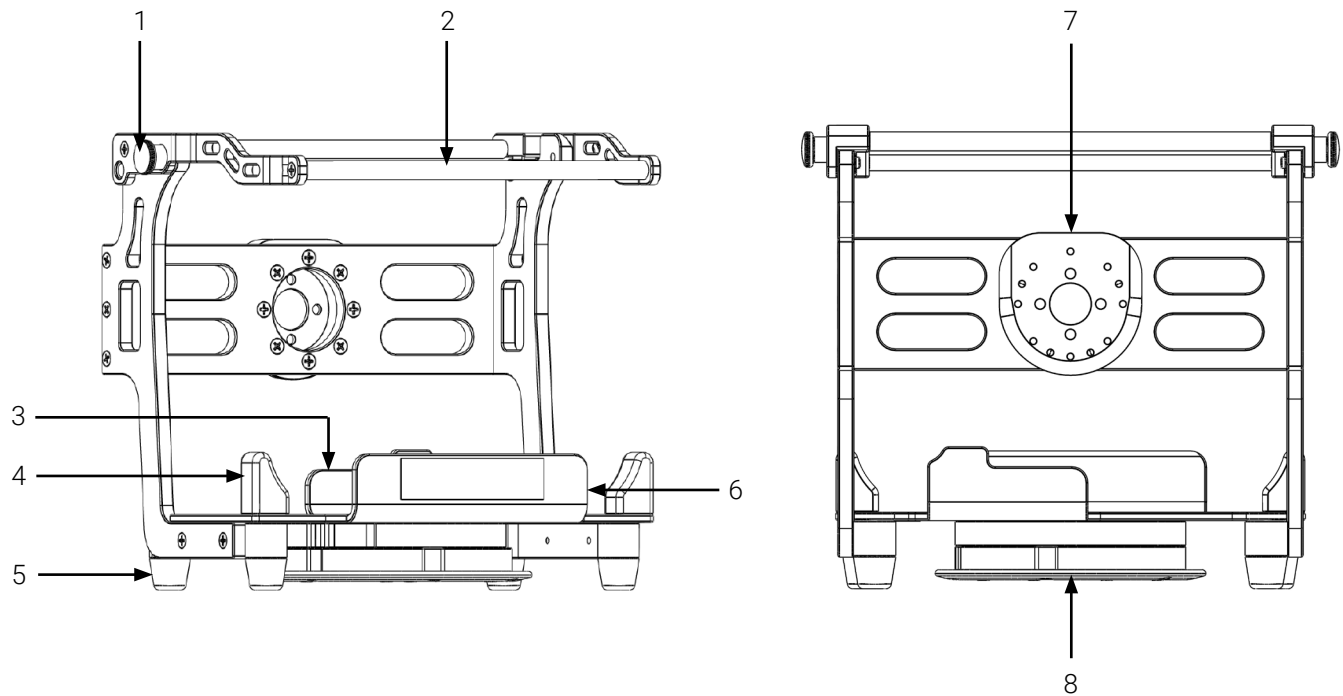

**Figure 2:** Bracket Pro Serie 280 illustrated parts (front and back of bracket shown)

## 6. Maintenance

Daily safety checks (refer to the Bracket Pro Serie 280 Operating Guide) and a condition-based maintenance plan (section 6.2) are required and should be established for all Technimount equipment. Factors such as weather, environment, geographical location, and individual usage will necessitate different needs. For the maintenance of the Bracket Pro Serie 280, follow the guidelines listed in the user manual, operating guide and, in accordance with your service's current maintenance practices and protocols.

**NOTE:** Always keep records of your all maintenance activities and immediately remove defective or expired products from your inventory.

Please contact Technical Support at [techsupport@technimount.com](mailto:techsupport@technimount.com) for replacement parts or repair related issues.



### **WARNING** - General warning

- **Do not** perform maintenance on Technimount products, before receiving proper training.
- Perform maintenance operations, as prescribed in the user manual. Failing to follow the recommended maintenance plan or its guidelines could cause premature damage to the product.
- Use the recommended parts, maintenance procedures, cleaning solutions and lubricants, as prescribed in the user manual. Using unapproved modified parts or procedures for the installation, operation, or maintenance of the Technimount product may cause the equipment to be unstable and could cause injury to the patients or EMS personnel and void the product warranty.
- Replace damaged or worn-out parts if past their expected service life or when damaged (refer to the Technical Specifications in Chapter 3 of this user manual). Recycle damaged parts or dispose according to the environmental laws that apply to your jurisdiction and consult the safety data sheets (SDS).



### **CAUTION** - Safe handling and operation

- **Do not** steam clean or use ultrasonic cleaners on the system or any of its components.
- **Do not** immerse the metal parts/components in water.
- To spot clean, the maximum water temperature should not exceed 180° F / 82° C. The maximum water pressure should not exceed 1500 psi / 103.5 BAR. If using a pressure washer to clean the unit, the pressure nozzle must be kept a minimum of 24 in. (61 cm) from the unit.
- Always rinse and dry the mounting systems properly after using cleaning products. Certain types of cleaners may leave a corrosive residue on the surface of the product and could cause the premature corrosion of critical components. Refer to the product Safety Data Sheets (SDS) for chemical information or handling, storage, and emergency measures in case of accident.



### **CAUTION** - Corrosion

- Some cleaning products are corrosive in nature and may cause damage to the product if used improperly. When cleaning, always use appropriate personal protection equipment (PPE) based on established protocols (e.g., gloves, eyewear, etc.).
- Dispose of corrosive wastes according to the environmental laws that apply to your jurisdiction and consult the safety data sheets (SDS).

## 6.1. Required Tools

- Clean dry cloth
- Soft brush
- Power washer
- Sodium Thiosulfate solution

## 6.2. Condition-Based Maintenance Plan

Condition-based maintenance is carried out by the maintenance supervisor when the safety checks have revealed underlining issues that need immediate attention, and / or to prolong the longevity of the system in optimal conditions. The maintenance supervisor should perform condition-based maintenance minimally every three (3) months, or when required. In case of a non-conformity, immediately stop using the product and contact Technical Support at [technicalsupport@technimount.com](mailto:technicalsupport@technimount.com) for a remedial action plan.

CONDITION-BASED MAINTENANCE		DONE	
TASKS		YES	NO
Perform the safety checks to get an overview of the system's overall current condition.		<input type="checkbox"/>	<input type="checkbox"/>
<b>Clean the System</b>		<input type="checkbox"/>	<input type="checkbox"/>
1. If needed, rough clean the standard bottom disc and anti-rotation micro disc using a soft brush to remove grease and dirt.			
2. Remove the contaminants collected during the use of the product using a power washer.			
3. Clean using a solution of 0.13 oz / 3.70 ml Sodium Thiosulfate in 1 pt. / 0.5 L of warm water and clean cloth. For stains, spot clean applying the solution directly ont the stain and let stand on the surface.			
<b>NOTE:</b> Avoid over saturation and ensure that the product does not stay wet longer than the cleaner's manufacturer guidelines for proper disinfecting.			
4. Thoroughly rinse the solution with clean water and clean cloth, then towel dry all the components before returning to service.			
<b>Inspect the Hardware</b>		<input type="checkbox"/>	<input type="checkbox"/>
1. Make sure all the screws are accounted for and that they are levelled with the surface in which they are screwed.			
2. Make sure there are no loose screws.			



**DONE**

Comments and observations:

## 7. Spare Parts & Upgrade Kits

Technimount reserves the right to change part numbers and products without notice. Please contact Customer Service at [customerservice@technimount.com](mailto:customerservice@technimount.com) to ensure product options and availability, or Technical Support at [techsupport@technimount.com](mailto:techsupport@technimount.com) for replacement parts or repair related issues.

PART / KIT NUMBER	PART / KIT DESCRIPTION
923-00-75-IND <sup>2</sup>	Bracket Pro Serie 280 replacement foot 3/4" (hardware included)
Contact Customer Service	Bracket Pro Serie mounting systems

---

<sup>2</sup> Installation requires red Loctite #262







# TECHNIMOUNT

EMS™

Technimount EMS offers mounting solutions that can be installed on ambulance counters, walls and stretchers which allows for the equipment to follow the patient throughout the continuum of care. Our unparalleled level of flexibility allows for maximum operativity in EMS, hospital and military environments.

Technimount EMS is driven to offer innovative solutions that respond to the unique device management needs of emergency and Critical Care Transport (CCT) teams for ground and air ambulances. Safety is at the core of our values, all Technimount systems are tested in compliance with the highest industry standards for impact resistance. Technimount EMS is committed to evolving its solutions as healthcare practices evolve.

**TECHNIMOUNT E.M.S. HOLDING INC.**

3770 Jean Marchand Street, Suite 100-C  
Quebec (QC) G2C 1Y6  
Canada  
T + 1 581.700.6613  
TF + 1 888.639.2758

[technimount.com](http://technimount.com)

