

Intended Use

This quick installation procedure is intended to assist you with the installation of the Universal Mounting Base™ on top of the emergency cart defibrillator mount arm.

Image references within this document may differ from actual product. For more information or to ensure product options and availability, please contact Customer Service at customerservice@technimount.com.

Technimount Medical™ Inc.

3770 Jean Marchand Street, Suite 100-B

Quebec (QC) G2C 1Y6

Canada

www.technimount.com

T + 1 581.700.6583 or TF + 1 888.521.0871 (North America)

Unpack the Universal Mounting Base™ and Support Plate

1. Inspect the shipping box(es) for signs of damage, before accepting shipment. Take pictures and report them promptly if applicable.
2. Move the shipping box(es) to the location of the installation.
3. Open and unpack the box(es). Ensure that all shipping and packaging materials have been properly removed. Keep all packaging material for future use.
4. Identify all the components and hardware included for the installation, then set aside (refer to Required Parts).
5. Inspect the items for signs of damage. Take pictures and report them if applicable.

Safety Measures



WARNING - Risk of Injury

Regulations and standards for safety are the sole responsibility of the end user. Ensure that the installation specifications meet the local and regional compliance requirements before use. Product incompatibility could cause unpredictable functioning resulting in injury to the patients or healthcare personnel.



WARNING - Part Breakage

Do not use powered tools to fasten the hardware during installation, as there is a potential risk of damage to the threads.



CAUTION - Follow Instructions for Use

- Always read and abide by all the safety guidelines identified and follow the instructions provided in this document.
- Refer to the crash cart's user manual for installation, safety precautions, maintenance and user instructions for the safe use of the cart.



CAUTION - Safe Handling and Operation

- **Do not** exceed the Safe Working Load (SWL) capacity of the base and/or cart.
- **Do not** use for transport in a vehicle. This product is intended only for indoor use in a clinical setting.
- **Do not** modify the product or any of its components and install only as described in this procedure. Modifying the product or using parts that have not been approved by Technimount Medical could cause unpredictable functioning, resulting in injury to the patients or healthcare personnel.
- Improper installation, maintenance, failure to properly clean or dispose of corrosive products may increase the risk of premature damage, impair the device, or cause injury to the patient or healthcare personnel.
- The administrator is responsible for providing proper training to any healthcare personnel who will install, operate and perform maintenance on Technimount products.
- If any serious incident occurs with the mounting solution, immediately stop using the product, report this incident to Technical Support at technicalsupport@technimount.com and the applicable regulatory agency.

Required Parts & Tools

- Universal Mounting Base™
- Support plate
- Philips flat head screws, 8-32 X 2 1/2 in. (3X)
- Philips pan head screws, 10-32 X 1/2 in. (4X)
- Nylon-insert locknuts (3X)
- Philips screwdriver #2
- Wrench, 11-32 in.

NOTE: The screwdriver type depends on the installation location. Refer to your maintenance department for more information.

Install the Universal Mounting Base™

1. Remove the defibrillator tray from the Avalo[®] emergency cart defibrillator mount arm (3 screws and 3 nuts; Figure 1 A).
2. Install the support plate on the defibrillator mount arm (3 flat head screws and 3 locknuts; Figure 1 B).
3. Install the Universal Mounting Base™ on the support plate (4 pan head screws; Figure 1 C).
4. Ensure proper functioning of the Universal Mounting Base™ blue quick release mechanism (Figure 1 D).

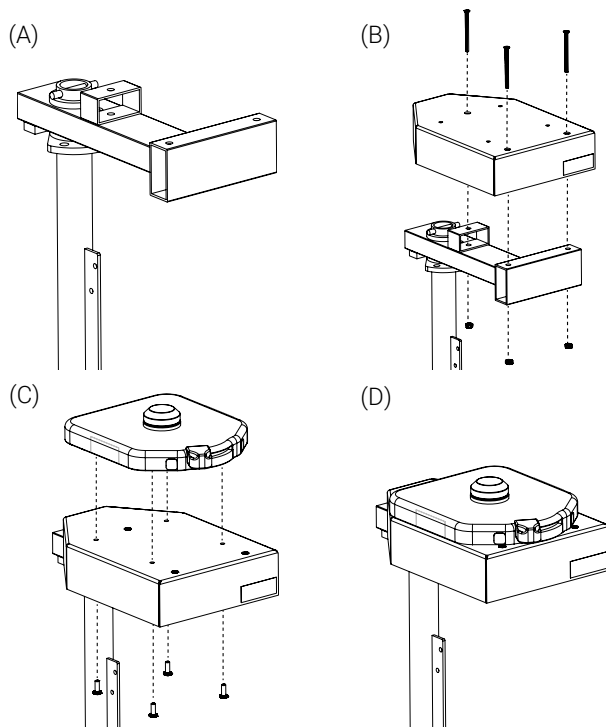


Figure 1: Universal Mounting Base™ installation

Operate the Universal Mounting Base™

To Install a Medical Device:

1. Ensure that the blue quick release mechanism is pushed all the way towards the left, in the unlocked position (Figure 2 A).
2. Align to center and place the medical device onto the Universal Mounting Base™ (Figure 2 A).
3. Slide the blue quick release mechanism towards the right (Figure 2 B) to lock and secure the medical device in the base.
4. Turn the medical device clockwise or counterclockwise, to the desired position.

To Remove a Medical Device:

1. Slide the blue quick release mechanism towards the left (Figure 2 C) to unlock the medical device from the base.
2. Lift to remove the medical device from the base and set aside (Figure 2 D).

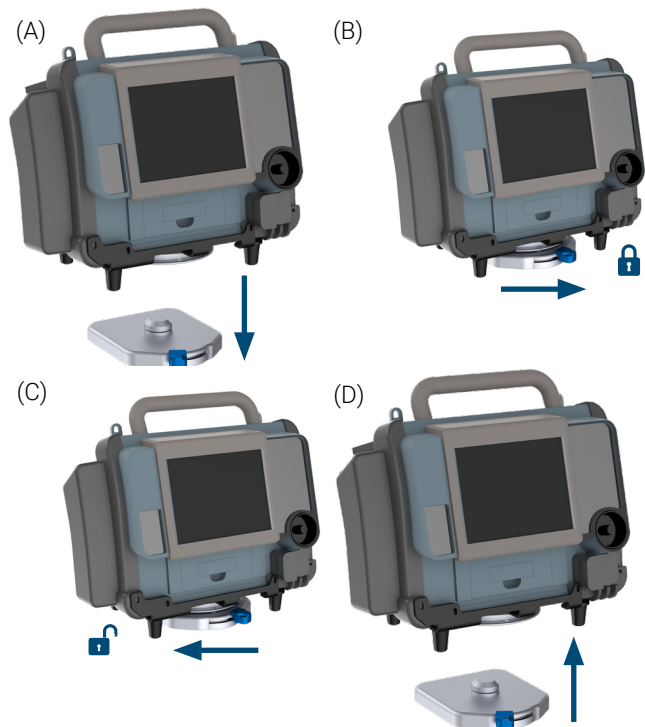


Figure 2: Operating the Universal Mounting Base™

¹ Product and medical device manufacturer names are Trademarks™ or Registered Trademarks® of their respective holders. Technimount does not have a commercial relationship with these medical device manufacturers.