




**TECHNIMOUNT**  
EMS™

# **BRACKET PRO SERIE® 92-FL**

## USER GUIDE

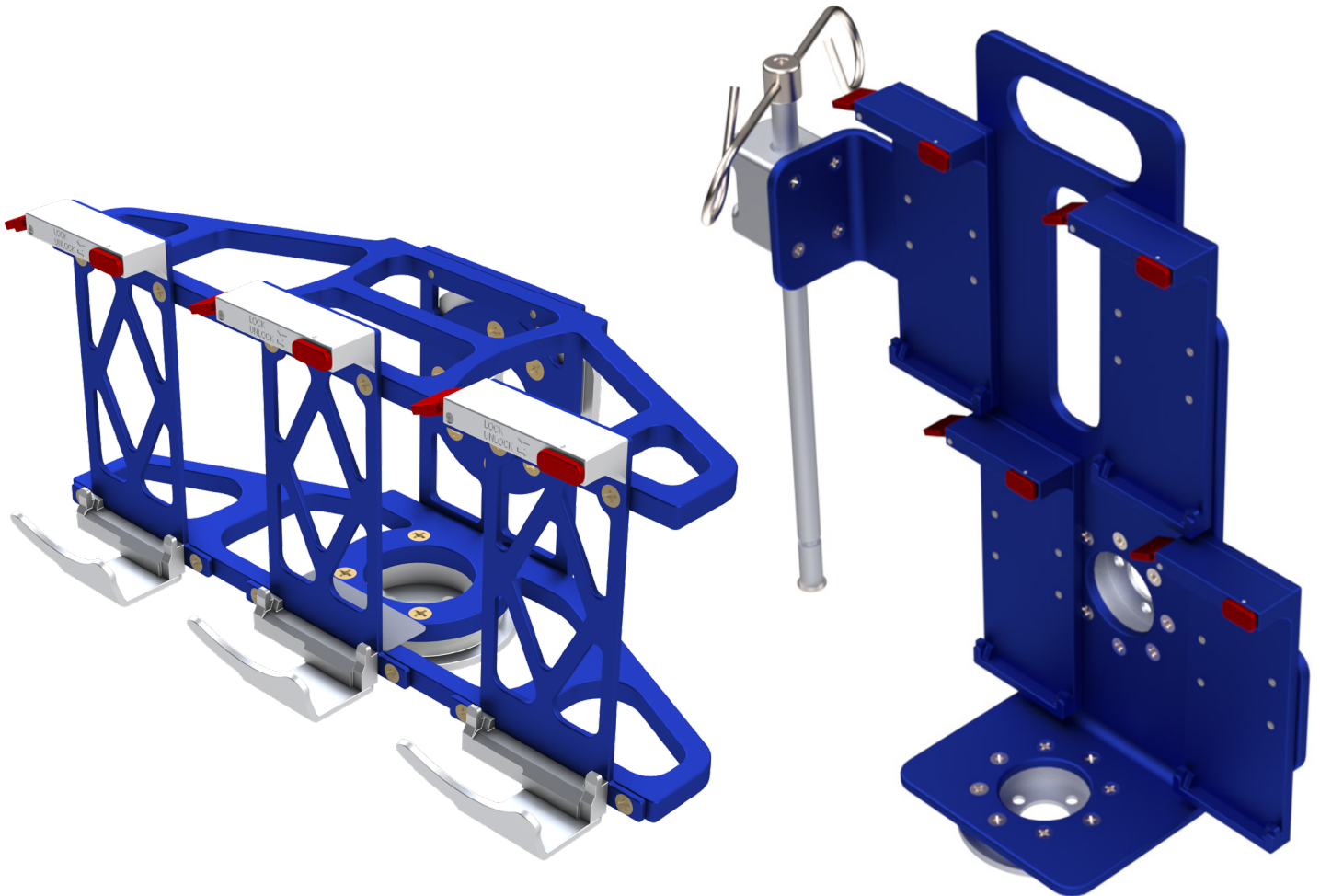

## COPYRIGHT

---

Copyright Technimount E.M.S. Holding Inc. (Technimount), an affiliated company of Technologies CGC Inc. All Rights Reserved. No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system, or translated into any language in any form or by any means without the written permission of Technimount or its affiliate companies.

### Trademark Attributions:

Technimount System™, Technimount EMS™ and Technimount Medical™ are registered trade marks of Technologies CGC Inc.

Bracket Pro Serie® is a registered trademark of Technologies CGC Inc.

Sapphire™ is a registered trademark of Eitan Medical.

In order to ensure continuous improvements to our products and services, we invite you to communicate your comments to our Customer Service Department at [customerservice@technimount.com](mailto:customerservice@technimount.com).

# CONTACT INFORMATION

---

## OFFICE ADDRESS

TECHNIMOUNT E.M.S. HOLDING INC.  
3770 Jean Marchand Street, Suite 100-C  
Quebec (QC) G2C 1Y6  
Canada

[info@technimount.com](mailto:info@technimount.com)

[www.technimount.com](http://www.technimount.com)

T + 1 581-700-6613


TF + 1 888.639.2758

## NOTE:

For any issues with your Technimount product, its components, or for any technical questions during the installation or maintenance, please send an Email to [techsupport@technimount.com](mailto:techsupport@technimount.com).

Please have the serial number of your Technimount product available (as shown in the figure below) when calling Technimount Customer Service or Technical Support. Include the serial number in all written communications.

## SERIAL NUMBER LOCATION

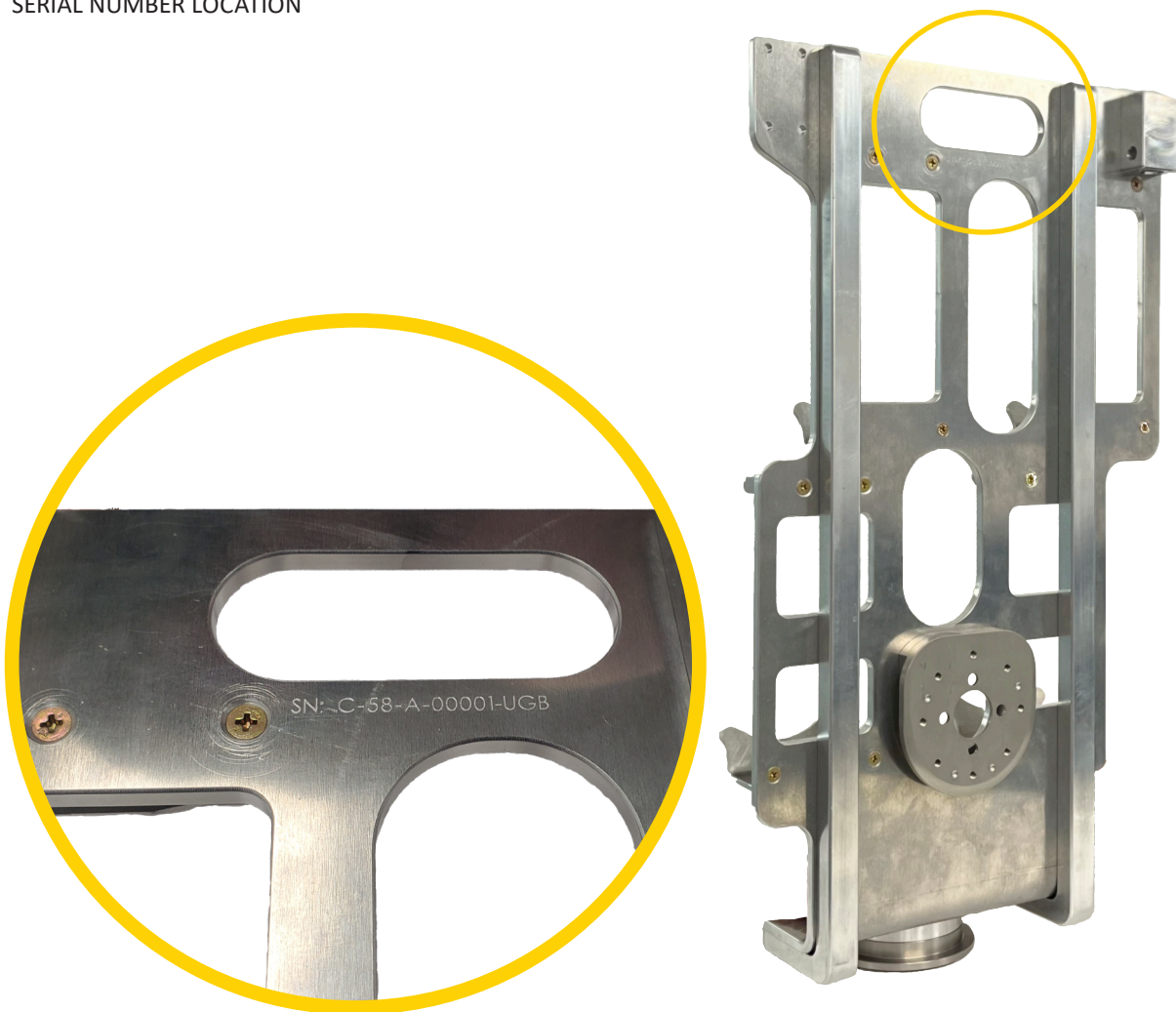

[Return To Table of Contents](#)

# TABLE OF CONTENTS

---

<b>COPYRIGHT .....</b>	<b>2</b>
<b>CONTACT INFORMATION.....</b>	<b>3</b>
<b>INTRODUCTION .....</b>	<b>6</b>
PURPOSE OF THE DOCUMENT .....	6
PRODUCT DESCRIPTION .....	6
INTENDED USE OF THE PRODUCT .....	6
POSSIBLE MOUNTING OPTIONS FOR THE BRACKET PRO SERIE® 92-FL .....	7
SYMBOLS AND DEFINITIONS .....	8
WARNING / CAUTION / NOTE.....	9
<i>BRACKET PRO SERIE 92-FL3</i> .....	10
<i>FRONT VIEW</i> .....	10
<i>BACK VIEW</i> .....	11
PRODUCT ILLUSTRATION.....	12
<i>BRACKET PRO SERIE 92-FL4</i> .....	12
<i>FRONT VIEW</i> .....	12
<i>BACK VIEW</i> .....	13
SPECIFICATIONS.....	14
NOTICE .....	15
<b>SUMMARY OF SAFETY PRECAUTIONS .....</b>	<b>16</b>
WARNING/CAUTION/NOTE .....	16
<b>INSTALLATION GUIDE .....</b>	<b>18</b>
REMOVING THE PACKAGING FOR INSTALLATION .....	18
TOOLS REQUIRED FOR INSTALLATION.....	18
INSTALLATION OF THE SAPPHIRE PUMPS ONTO THE BRACKET .....	19
REMOVING THE SAPPHIRE PUMPS FROM THE BRACKET .....	22
<b>OPERATION GUIDE .....</b>	<b>24</b>
INSTALLING THE BP92-FL4 BRACKET ONTO A HORIZONTAL MICRO BASE.....	24
REMOVING THE BP92-FL4 BRACKET FROM A HORIZONTAL MICRO BASE .....	25
INSTALLING THE BP92-FL3 BRACKET ONTO A HORIZONTAL MICRO BASE.....	26
REMOVING THE BP92-FL3 BRACKET FROM A HORIZONTAL MICRO BASE .....	27
GENERAL OPERATING GUIDELINES FOR THE SAFETY MD TRANSPORTER .....	28
INSTALLATION AND REMOVAL OF THE BRACKET ONTO AND FROM THE SAFETY MD TRANSPORTER VERTICAL MICRO BASE .....	29
<b>MOUNTING SYSTEM PART NUMBERS.....</b>	<b>30</b>
KITS AND OTHER OPTIONS .....	30
<b>MAINTENANCE GUIDE.....</b>	<b>31</b>
CLEANING .....	31
<i>CLEANING PROCESS</i> .....	31
<i>CLEANING SOLUTIONS</i> .....	32
<i>REMOVAL OF IODINE COMPOUNDS</i> .....	32
PREVENTIVE MAINTENANCE.....	32
INSPECTION PROCESS AND SCHEDULE.....	33
<i>MAINTENANCE PROGRAM</i> .....	33
INSPECTION AND MAINTENANCE RECORD.....	34
TRAINING RECORD.....	35

# TABLE OF CONTENTS

---

REPLACEMENT PARTS.....	36
<b>WARRANTY .....</b>	<b>37</b>
WARRANTY POLICY .....	37
LIMITED RESPONSIBILITY AND WARRANTY.....	37
INTERNATIONAL WARRANTY CLAUSE .....	37
RETURN POLICY.....	38
<i>PRIOR TO 30 DAYS</i> .....	38
<i>PRIOR 45 DAYS</i> .....	38
<i>PRIOR 60 DAYS</i> .....	38
RETURN AUTHORIZATION.....	38
DAMAGED MERCHANDISE.....	39
CLAIM PROCESS .....	39
<i>REQUIRED INFORMATION</i> .....	39
<i>FINDINGS AND CONCLUSION</i> .....	39
QUESTIONS ABOUT OUR POLICY .....	39
<b>SAE CERTIFICATION .....</b>	<b>40</b>

# INTRODUCTION

---

## PURPOSE OF THE DOCUMENT

This User Guide is designed to assist you with the installation, operation and maintenance of the Technimount Bracket Pro Serie 92-FL that was designed for the Sapphire™ - Medical Infusion Pump for Infusion Therapy.

## PRODUCT DESCRIPTION

The Bracket Pro Serie 92-FL is a complete mounting system for Sapphire™ - Medical Infusion Pump which seamlessly transitions from aircraft to ground ambulance to hospital, following the patient during care. The lightweight frame of the mounting system is made from aluminum which is resistant for applications that require extreme durability. As the most flexible solution available on the market, the Bracket Pro Serie 92-FL is available in two options for flight (3 and 4 pumps). It can fit any ground and air ambulance configurations in North American and European markets. These solutions provide a higher flexibility for different EMS/CCT protocols and applications.

## INTENDED USE OF THE PRODUCT

The Bracket Pro Serie 92-FL is a mounting solution to securely transport the Sapphire™ - Medical Infusion Pump during EMS and Critical Care Transport. It is designed to be installed on the equipment as described in this user guide. This equipment is for crew who has received the appropriate training necessary to operate the equipment in the field according to its intended use as outlined in this user guide. Please read this User Guide thoroughly before using this equipment. To ensure the safe operation of this equipment, it is recommended to train staff prior to use.



# INTRODUCTION

## POSSIBLE MOUNTING OPTIONS FOR THE BRACKET PRO SERIE® 92-FL


Safety MD Transporter  
with Vertical Micro Base



XTPA-LP with Horizontal  
Micro Base

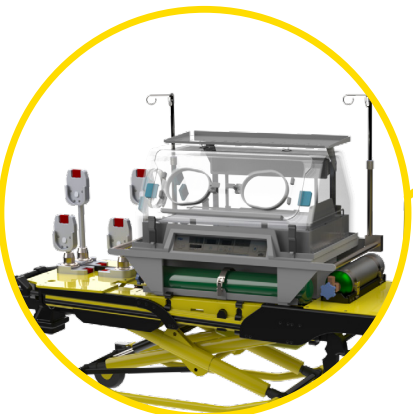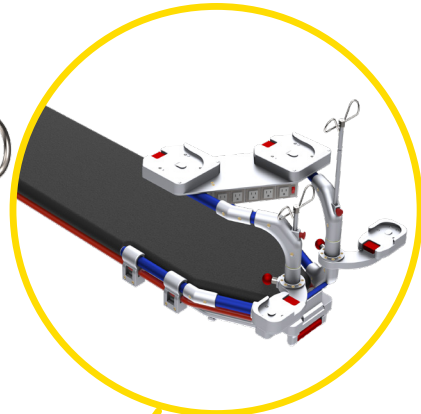

Neonatal Transport  
System with Vertical  
Micro Bases

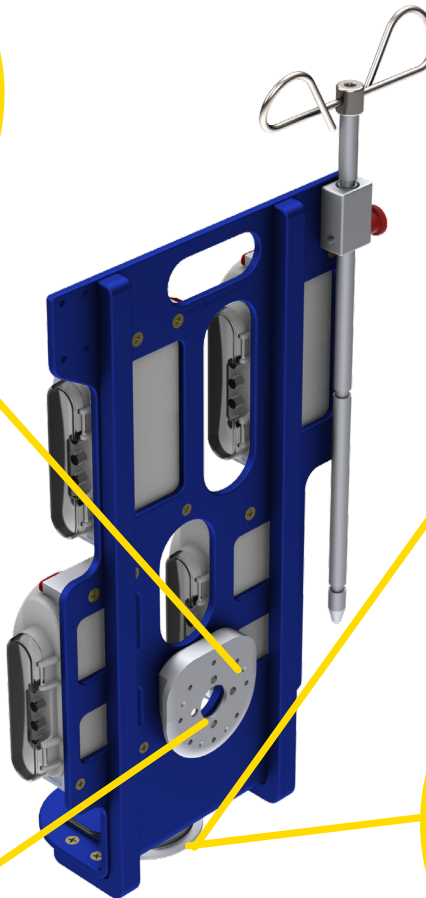




Safety MD-  
Transporter® with  
Horizontal Micro  
Base

[Return To Table of Contents](#)

# INTRODUCTION

---

## SYMBOLS AND DEFINITIONS

<u>SYMBOL</u>	<u>DESCRIPTION</u>
	Warning and Caution, special attention is required. Consult accompanying documents
	Safe working load symbol and Load balance symbol
	Pinch Point





# INTRODUCTION

---

## WARNING / CAUTION / NOTE

The word warning, caution, or note carry special meaning and should be carefully reviewed.

<u>SYMBOL</u>	<u>DESCRIPTION</u>
	<p><b>WARNING</b></p> <p>Alerts the reader about a situation which, if not avoided, could result in death or serious injury. It may also describe potential serious adverse reactions and safety hazards.</p>
	<p><b>CAUTION</b></p> <p>Alerts the reader of a potentially hazardous situation, which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property. This includes special care necessary for the safe and effective use of the device to avoid damage that may occur as a result of use or misuse.</p>
<b>NOTE</b>	<p>Provides special information about the product.</p>


# INTRODUCTION

---

## PRODUCT ILLUSTRATION (CONTINUED)

[BRACKET PRO SERIE 92-FL3](#)

[Front View](#)

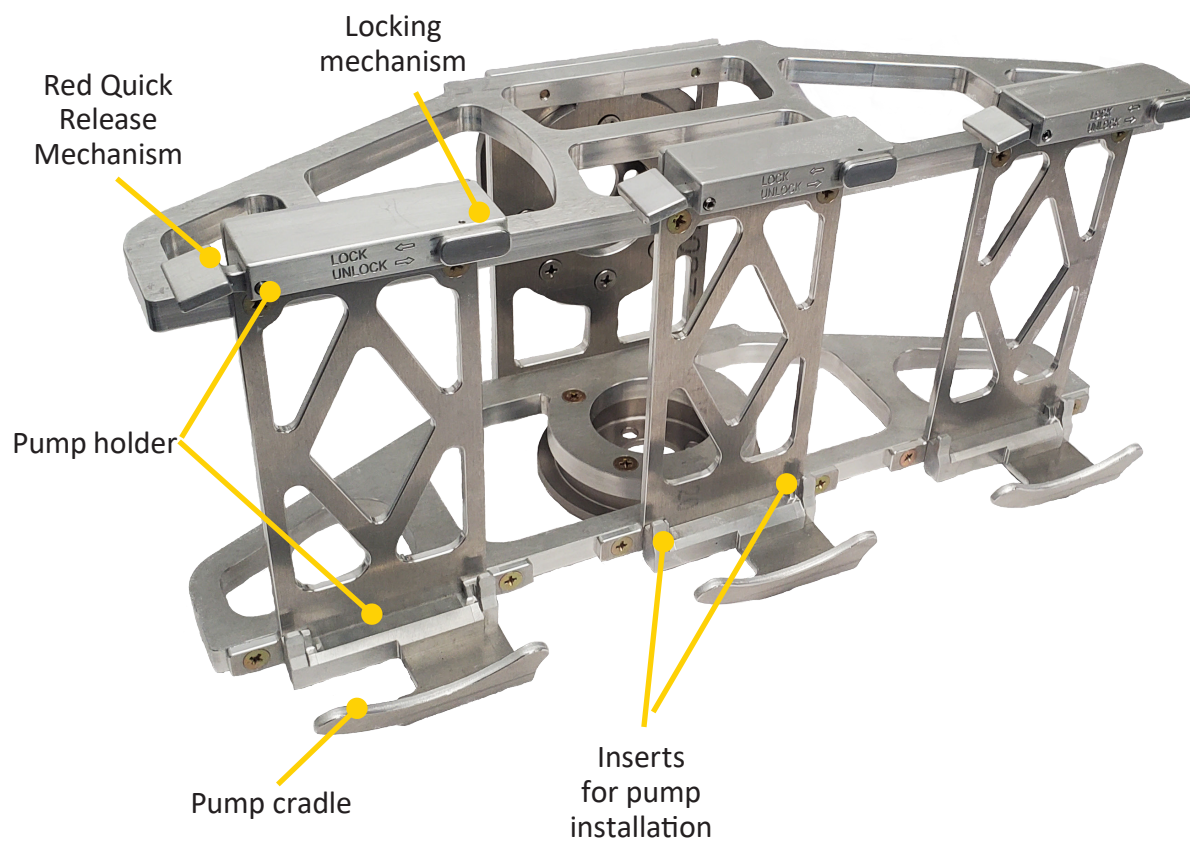

[Return To Table of Contents](#)


# INTRODUCTION

---

## PRODUCT ILLUSTRATION (CONTINUED)

### [BRACKET PRO SERIE 92-FL3 \(CONTINUED\)](#)

#### [Back View](#)

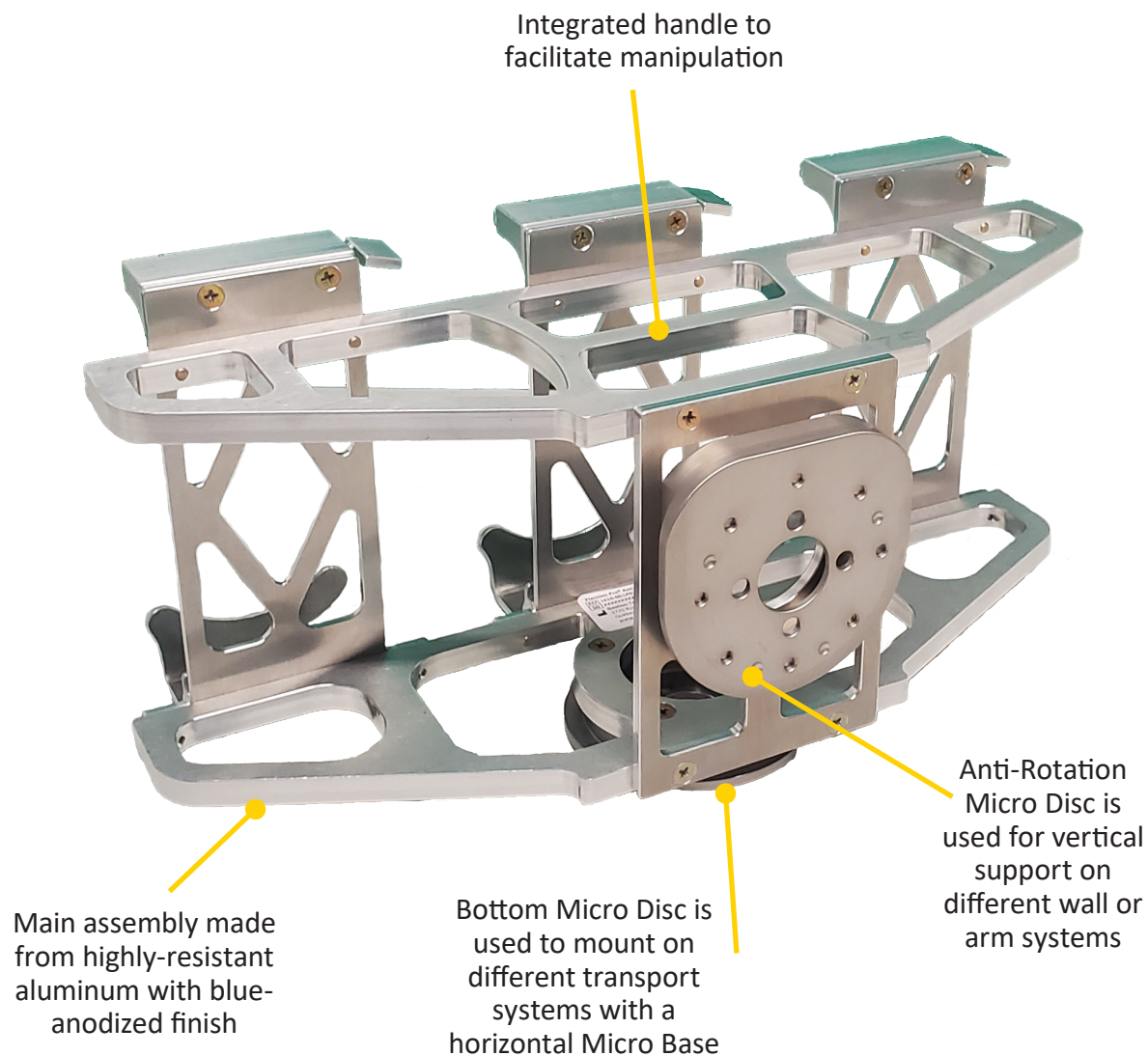


[Return To Table of Contents](#)

# INTRODUCTION

## PRODUCT ILLUSTRATION

[BRACKET PRO SERIE 92-FL4](#)

[Front View](#)

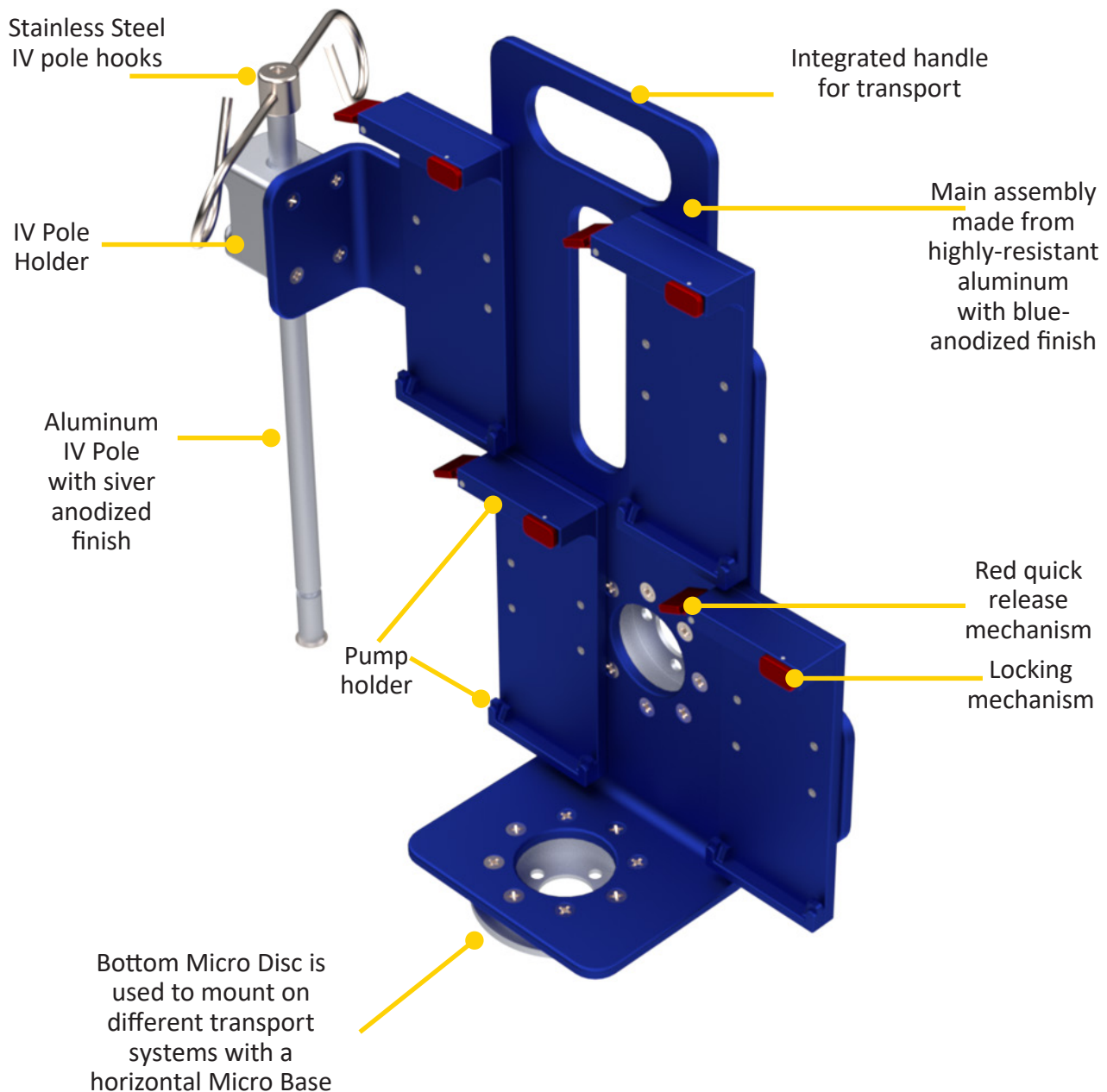


[Return To Table of Contents](#)

# INTRODUCTION

## PRODUCT ILLUSTRATION (CONTINUED)

### BRACKET PRO SERIE 92-FL4 (CONTINUED)

#### Back View

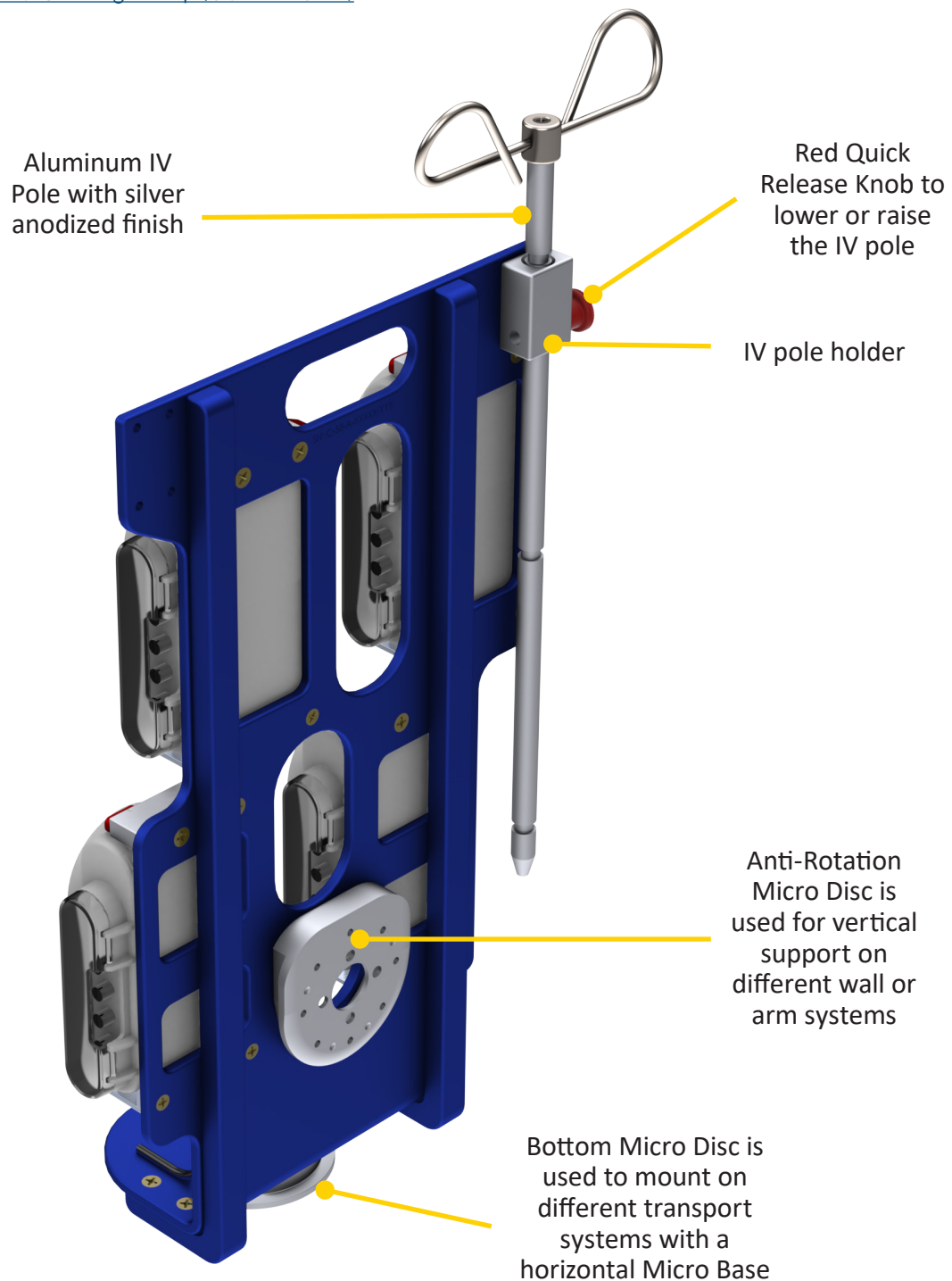

[Return To Table of Contents](#)

# INTRODUCTION

## SPECIFICATIONS

<b>Product Name</b>	Bracket Pro Serie® 92-FL3 Bracket Pro Serie® 92-FL4
<b>Product Description</b>	Bracket mounting system to secure the Sapphire Pumps on surfaces, cots/ stretchers and walls in emergency vehicles
<b>Compatibility of Medical Device*</b>	Sapphire™ - Medical Infusion Pump for Infusion Therapy
<b>Environment of Usage</b>	EMS/CCT for Air Transport
<b>Product Section Material</b>	<ul style="list-style-type: none"><li>Aluminum 6061-T6</li><li>Stainless Steel</li></ul>
<b>Product Finish</b>	<ul style="list-style-type: none"><li>Main Assembly: Blue-Anodized Finish</li><li>Bottom Micro Disc: Silver-Anodized Finish</li><li>Anti-Rotation Micro Disc: Silver-Anodized Finish</li><li>IV Pole: Silver-Anodized Finish</li><li>IV Pole Hook: Stainless Steel</li></ul>
<b>Temperature of Usage</b>	- 35° C to 45° C
<b>Certification</b>	Tested in Compliance with: <ul style="list-style-type: none"><li>SAE J3043</li><li>FAA 14 CFR § 23 (Applicable Sections)</li></ul>
<b>Installation</b>	Proprietary Locking Mechanism on Each Pump Bracket Holder
<b>Attachment Options</b>	2 Locking Discs – Bottom Micro Disc – Anti-Rotation Micro Disc
<b>Transportation Compatibility System</b>	Compatible with Technimount's: <ul style="list-style-type: none"><li>Wall Mount Systems</li><li>Safety MD-Transporters</li><li>Horizontal and Vertical Micro Bases installed on Various Technimount Mounting Solutions</li></ul>
<b>Features</b>	<ul style="list-style-type: none"><li>Robust Mounting Solution for EMS/CCT/ALS</li><li>Air &amp; Ground Compatibility</li><li>Red Quick Release Mechanism to Easily Insert &amp; Remove Pumps from Bracket Holder</li><li>Two Discs (Back Anti-Rotation Micro Disc and Bottom Micro Disc) for different mounting systems</li></ul>
<b>Physical Overall Dimensions</b>	Bracket Pro Serie 92-FL3 <ul style="list-style-type: none"><li>Lenght 13.625 in, (33.02 cm) Depth 6.516 in (15.24 cm), Height 6.175 in. (15.24 cm)</li></ul> Bracket Pro Serie 92-FL4 <ul style="list-style-type: none"><li>Lenght 13.39 in, (33.02 cm) Depth 4.90 in (10.16 cm), Height 21.72 in. (53.34 cm)</li></ul>
<b>Weight</b>	<ul style="list-style-type: none"><li>Bracket Pro Serie 92-FL3 - 3.30 lb (01.5kg)</li><li>Bracket Pro Serie 92-FL4 - 11.15 lb (5.07 kg)</li></ul>
<b>Part Number</b>	<ul style="list-style-type: none"><li>2300-10-MTP-FL3 - Bracket Pro Serie 92-FL3</li><li>2300-10-MTP-FL4 - Bracket Pro Serie 92-FL4</li></ul>
<b>Other Models and Configurations</b>	Contact Technimount EMS for more information or for other options at <a href="mailto:customerservice@technimount.com">customerservice@technimount.com</a>

\* Product and medical equipment manufacturers names are trademarks™ or registered trademarks® of their respective holders. Technimount does not have a commercial relationship with these medical equipment manufacturers.

[Return To Table of Contents](#)

# INTRODUCTION

---

## NOTICE

Technimount hereby declares that the Bracket Pro Serie 92-FL3 or Bracket Pro Serie 92-FL4 Mounting Systems are in compliance with the essential requirements and other relevant provisions of the SAE J3043 and FAA 14 CFR § 23 (Applicable Sections) . For more information, please contact Technimount Customer Service ([customerservice@technimount.com](mailto:customerservice@technimount.com)).

TECHNIMOUNT E.M.S. HOLDING INC.

C/O: Regulatory Affairs

3770 Jean Marchand Street, Suite 100-C

Quebec (QC) G2C 1Y6

Canada

T + 1 581-700-6613

TF + 1 888.639.2758

[technimount.com](http://technimount.com)

[info@technimount.com](mailto:info@technimount.com)



## WARNING

The end user and administrator are responsible to validate regulations and standards for safety in their region, in order to comply with applicable safety regulations. Technimount is not responsible to inform the end user and/or the administrator of applicable legislation for safety in their area.

# SUMMARY OF SAFETY PRECAUTIONS

---

## WARNING/CAUTION/NOTE

Carefully read and strictly follow all Warnings and Cautions listed in this user guide. Servicing and maintenance should only be done by qualified personnel.



### WARNING

---

- ✓ Always hold the Bracket with both hands when you manipulate it.
- ✓ Operate the bracket only as described in this user guide.
- ✓ Do not modify the Bracket, or any components of Technimount systems. Modifying the product can cause unpredictable operation resulting in injury to the patient or operator.
- ✓ It is the responsibility of the operator of the Bracket or any of its options, to ensure equipment being used with the Technimount products meets the installation specifications. Injury may result if non-compatible products are used with Technimount EMS products.
- ✓ Do not attempt to operate, install or remove the equipment while mobile. Ensure that installation and removal of equipment is done while vehicle is immobilized.
- ✓ Verify that the bracket is inserted correctly into a mounting system and locked. If equipment is improperly installed and locked, it can cause the equipment to fall and cause injury to the patient or operator and/or damage to the medical equipment.
- ✓ Do not allow untrained staff to assist in the operation of the mounting system. Untrained technicians/staff can cause injury to the patient or themselves.
- ✓ Do not place additional equipment or items onto the Bracket or mounts other than approved Technimount systems or components, as this could damage the product and may result in injury to the patient or operator.
- ✓ Operators must be able to safely lift the total weight of the Bracket System and Medical Device.
- ✓ Do not attach restraints to the Bracket frame. It may result in damage to the System and/or result in injury to the patient or operator.
- ✓ Always reposition the equipment with care, avoid any improper pushing or violent manipulation, as it may result in unbalanced equipment and may cause injury to the patient or operator and/or damage to the product.
- ✓ Improper maintenance can cause injury to patient or operator or damage to the product. Maintain the Bracket and Mounting Systems as described in this user guide. Use only Technimount approved parts and maintenance procedures. Using unapproved parts and procedures could cause unpredictable operation and/or injury and will void the product warranty.
- ✓ Failure to properly clean or dispose of corrosive products will increase the risk of premature damage and may cause injury to the patient or operator.
- ✓ Any installation on a surface, wall or on the structure of the vehicle or the aircraft has to be validated or inspected in order to ensure it maintains the integrity of the vehicle or the aircraft.



# SUMMARY OF SAFETY PRECAUTIONS

---

## WARNING/CAUTION/NOTE (CONTINUED)

---



### CAUTION

---

- ✓ Only certified technical personnel familiar with the Bracket installation and use should manipulate it. Always refer to Technimount's user guides for all instructions for installing and using any Technimount Mounting System.
- ✓ Inspect regularly all components on the equipment for any issue and loose screws, bolts and nuts.
- ✓ Never install the mount into or on other similar mounting systems or brackets without written confirmation by Technimount System as these different systems may be from other brands or models and may not be compatible with Technimount Systems
- ✓ Fluid Bag Pole is meant to be used only for the temporary transfer between the hospital and ambulance and during air transport. This pole CANNOT be used inside the ground ambulance during transport.

### NOTE

- ✓ Some cleaning products are corrosive in nature and may cause damage to the product if used improperly. If such products are used to clean Technimount equipment, the equipment must be rinsed with clean water and thoroughly dried following the cleaning. Failure to properly rinse and dry the Mounting Systems will leave a corrosive residue on the surface, possibly causing premature corrosion of critical components.
- ✓ When ordering, it is important to identify on which mounting system you need to mount the bracket on as their are different systems for different EMS applications, e.g., EMS ground mounting systems.
- ✓ A maintenance program should be established for all Technimount equipment. Preventive maintenance may need to be performed more frequently, based on the level of use of the product. Close attention should be given to safety features (i.e.: screws, quick release mechanisms, and attachment points).
- ✓ Failure to use authorized parts, and lubricants, etc., could cause damage to the equipment and will void the warranty of the product.
- ✓ This user guide should be considered a permanent part of the Bracket, and should remain with the product even if the equipment is subsequently sold.
- ✓ Technimount continually seeks advancements in product design and quality. Therefore, while this user guide contains the most current product information available at the time of printing, there may contain minor differences between your mounting system and this user guide. If you have any questions, please contact Technimount Customer Service or Technical Support at [customerservice@technimount.com](mailto:customerservice@technimount.com) +1 888.639.2758

# INSTALLATION GUIDE

---

## REMOVING THE PACKAGING FOR INSTALLATION

1. Unpack boxes and check all items.
2. Ensure that all shipping and packaging materials have been properly removed from the product(s) prior to installation.
3. Identify all of the product components and hardware prior to starting the installation.
4. The Bracket System must work properly before being put into service.
5. Refer to the identification of components at the beginning of this user guide if needed.

## TOOLS REQUIRED FOR INSTALLATION


No tools are required for installation. If you have any questions regarding the installation, or need assistance, please contact our Technical Support team at [techsupport@technimount.com](mailto:techsupport@technimount.com)

# INSTALLATION GUIDE

---

## INSTALLATION OF THE SAPPHIRE PUMPS ONTO THE BRACKET

1. Locate the two holes on the bottom of the pump.

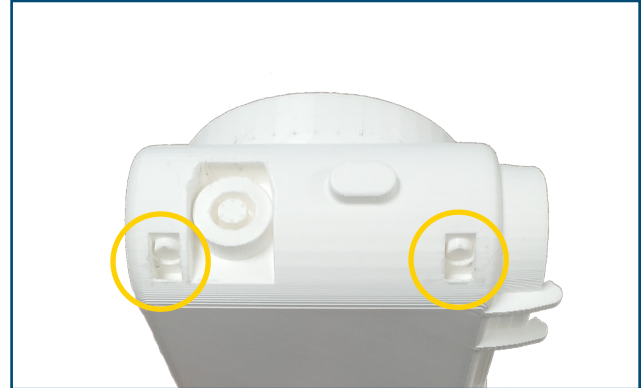


### NOTE:

To facilitate installation of pumps onto the bracket, it is recommended to install the bracket onto the horizontal micro base of a mounting system.



2. Locate inserts on the bottom of the the bracket's pump holders.

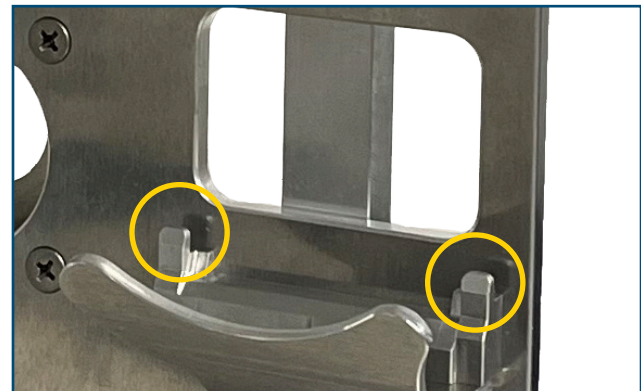

# INSTALLATION GUIDE

## INSTALLATION OF THE SAPPHIRE PUMPS ONTO THE BRACKET (CONTINUED)

3. Align the bottom of the pump with the bracket inserts on both sides.
4. Insert at a angle to facilitate installation.
5. Ensure that both sides are properly aligned with the bracket inserts.



6. The pump is installed when it is properly inserted and you have heard a soft clicking sound.



7. Slide the locking mechanism to the left to lock the pump securely into its bracket.



### WARNING

Always ensure that the pump is well locked by pulling on the pump. If it cannot be removed, that means it is properly locked. If not properly locked, the pump could fall out of its bracket during transport.



# INSTALLATION GUIDE

---

## INSTALLATION OF THE SAPPHIRE PUMPS ONTO THE BRACKET (CONTINUED)

8. Proceed to install all pumps while following steps 4 to 7.



### WARNING

Always check that all pumps are well locked by pulling on each pump. If it cannot be removed, that means it is properly locked.

9. Once properly locked the Sapphire Pumps are ready for EMS transport.




# INSTALLATION GUIDE

## REMOVING THE SAPPHIRE PUMPS FROM THE BRACKET

1. Slide right to unlock the pump from its bracket.



2. Press and hold the Red Quick Release Mechanism, to remove the pump from its bracket.

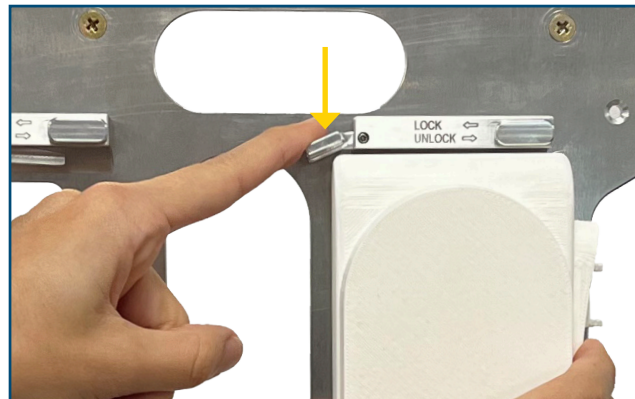


3. Hold the Red Quick Release Mechanism and slide pump outward using an angle to facilitate removal of the pump. Release mechanism once pump is removed.
4. Repeat steps 1 to 3 for each pump you want to remove from the bracket.




# INSTALLATION GUIDE

## INSTALLATION, REMOVAL AND USE OF THE IV POLE


1. Pull on Red Quick Release Knob.
2. Insert the IV pole from the top into the IV pole holder.
3. Slide in until you reach one of the grooves on the IV pole to hold into position.

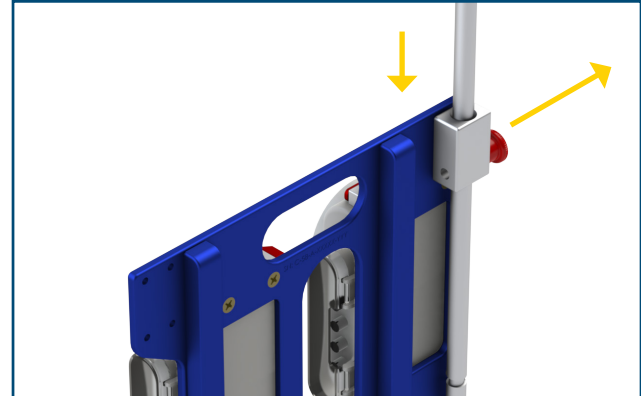

4. There are 2 positions for the IV Pole.
  - a. Lowest position when not in use.
  - b. Highest position when in use.
5. To lower or raise the IV pole, pull and hold the Quick Release Knob IV on the pole holder.
6. Lower or raise the IV pole until you reach the notch in the IV pole (in the highest or lowest position).
7. Release the Quick Release Knob once you reach the desired position.

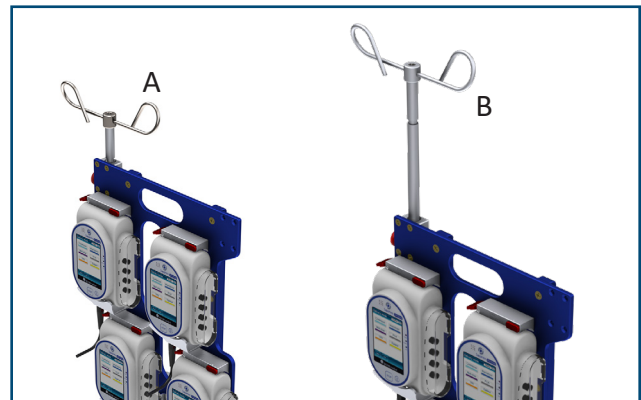


8. To remove the IV pole, pull on the Quick Release Knob and hold while removing the IV pole from its holder.

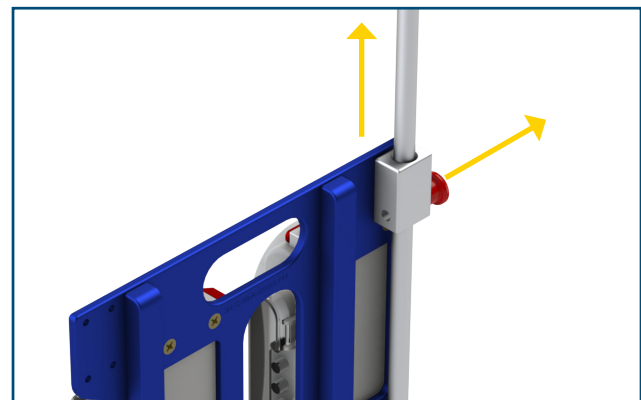




## INSTALLING THE BP92-FL4 BRACKET ONTO A HORIZONTAL MICRO BASE

1. Only Technimount brackets with a Bottom Micro Disc can be inserted into a horizontal Micro Base.

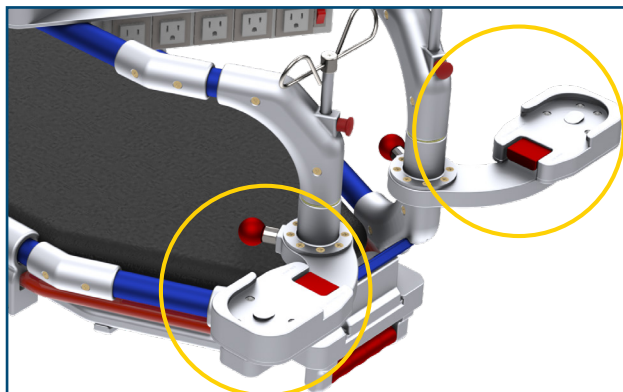

2. Ensure that your Technimount bracket has an Bottom Micro Disc to install the bracket onto the horizontal Micro Base.

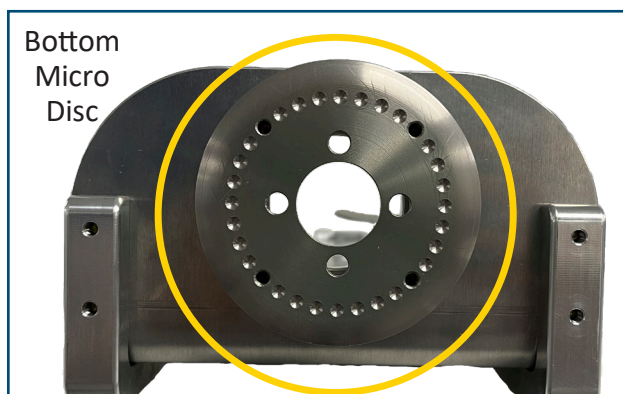


3. Not to create a lever effect, and to facilitate installation, it is recommended to position hands near the bottom of the bracket to install the bracket onto the Micro Base.
4. While using both hands, slowly insert the Bottom Micro Disc onto the Micro Base.



### WARNING


- ✓ Ensure that the Technimount bracket is securely locked into the Micro Base before transport.

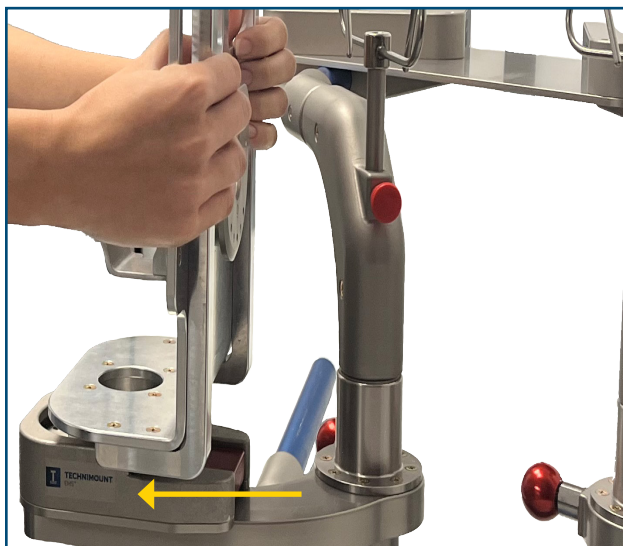




## REMOVING THE BP92-FL4 BRACKET FROM A HORIZONTAL MICRO BASE

1. To remove the bracket from the Micro Base, press the Red Quick Release Mechanism located on the front of the Micro Base.
2. While holding the Red Quick Release Mechanism with the thumb, place forefingers inside the bottom hole of the bracket as shown. Slide the bracket frontwards using a side to side motion to facilitate removal of the bracket from the Micro Base.
3. When the bracket is disengaged, release the Red Quick Release Mechanism and continue to slide frontwards until the bracket is removed.

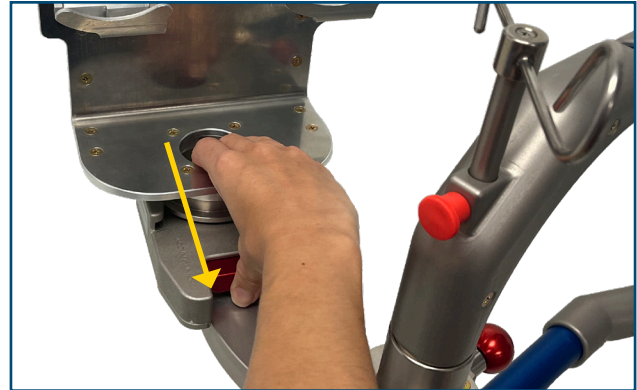


## INSTALLING THE BP92-FL3 BRACKET ONTO A HORIZONTAL MICRO BASE

1. Only Technimount brackets with a Bottom Micro Disc can be inserted into a horizontal Micro Base.

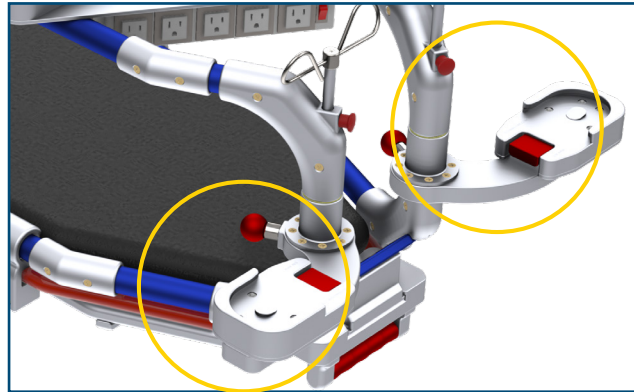

2. Ensure that your Technimount bracket has an Bottom Micro Disc to install the bracket onto the horizontal Micro Base.

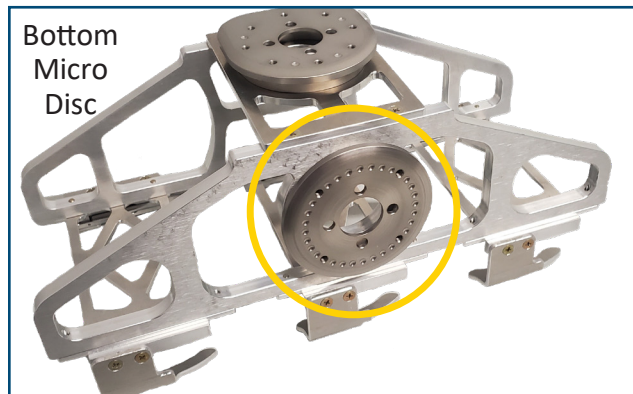


3. Install the BP92-FL3 bracket sideways as shown to facilitate installation onto the Micro Base.
4. While using both hands, slowly insert the Bottom Micro Disc onto the Micro Base all the way until you hear the locking sound.




### WARNING

- ✓ Ensure that the bracket is securely locked into the Micro Base before transport.

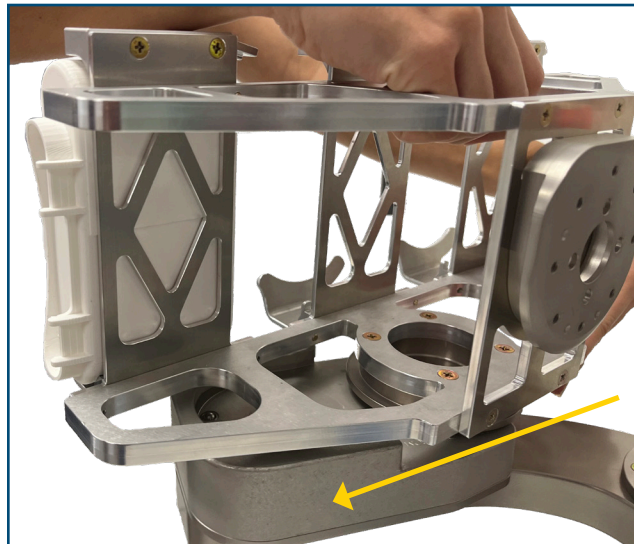


## REMOVING THE BP92-FL<sub>3</sub> BRACKET FROM A HORIZONTAL MICRO BASE


1. To remove the bracket from the Micro Base, press the Red Quick Release Mechanism located on the front of the Micro Base.
2. While holding the Red Quick Release Mechanism with the thumb, hold the top of the bracket as shown.

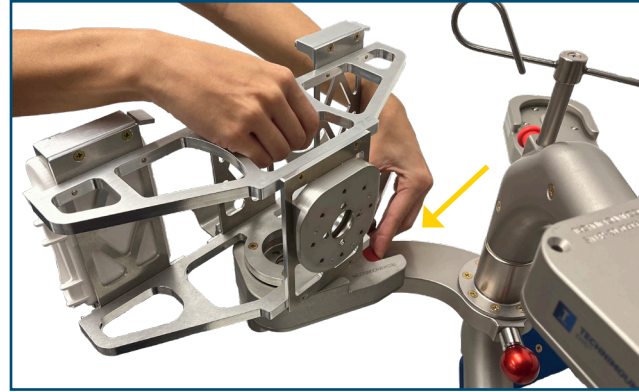

3. Slide the bracket frontwards using a side to side motion to disengage from the locking mechanism.

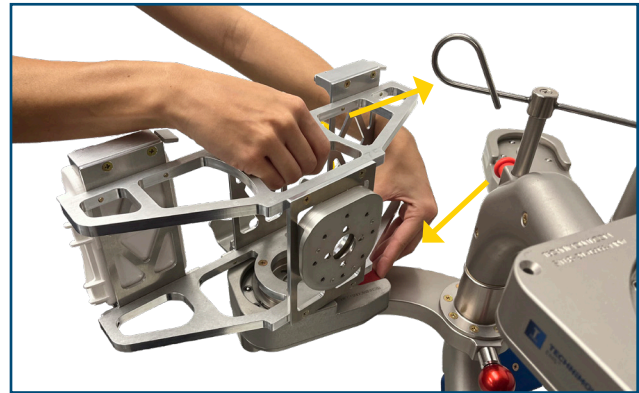

4. As soon as the bracket is disengaged from the Micro Base Lock, release the Red Quick Release Mechanism and continue to slide frontwards until the bracket is removed.

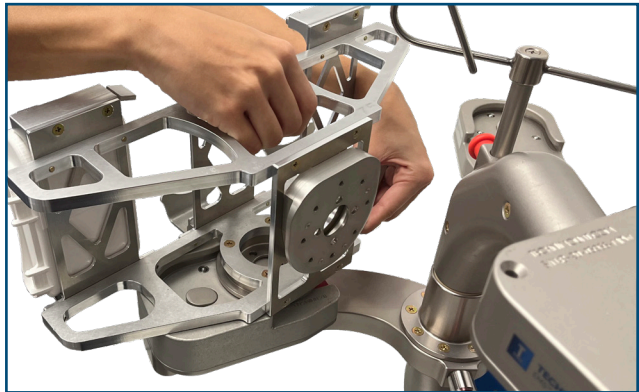

## GENERAL OPERATING GUIDELINES FOR THE SAFETY MD TRANSPORTER

- ✓ Use the Safety MD-Transporter only as described in its User Guide.
- ✓ Read all the labels and instructions on the product before using the Safety MD-Transporter.
- ✓ Always inspect the Micro Base and the Safety MD-Transporter for any issue or damage before every use.
- ✓ Installing or removing the Safety MD-Transporter requires a minimum of one (1) trained operator.
- ✓ Do not remove the Safety MD-Transporter while moving with the cot/stretcher.
- ✓ Stay with the patient and control the Safety MD-Transporter at all times.



---

### WARNING

- ✓ Never let a patient grab, manipulate or hold the Safety MD-Transporter, or the Micro Base on the cot/stretcher because it may cause injury to the operator or patient. Secure the Safety MD-Transporter as recommended while the patient is on the cot/stretcher.


### NOTE

The installation, rotating and removal procedures in the following section is demonstrated using a generic medical device in a Technimount bracket. Please follow the same procedures for your Technimount bracket.

For any questions, please contact Customer Service at [customerservice@technimount.com](mailto:customerservice@technimount.com).

### INSTALLATION AND REMOVAL OF THE BRACKET ONTO AND FROM THE SAFETY MD TRANSPORTER VERTICAL MICRO BASE


1. Only Technimount brackets with an Anti-Rotation Micro Disc on the back can be inserted into the Micro Base located on the Safety MD-Transporter.
2. Ensure that your bracket has an Anti-Rotation Micro Disc on the back to install the medical device onto the Safety MD-Transporter.

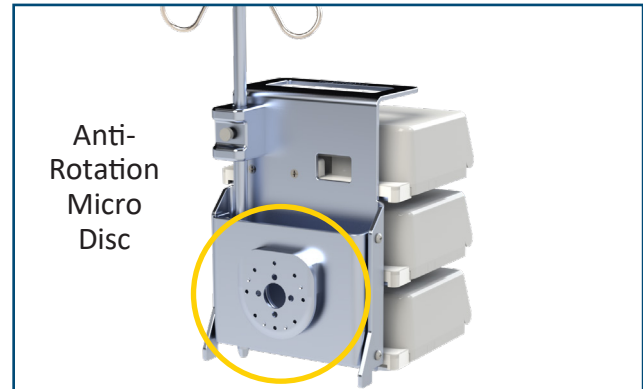

3. While using both hands, slowly insert the Anti-Rotation Micro Disc located on the back of the bracket into the Micro Base until you hear the locking sound.

#### WARNING

- ✓ Ensure that the bracket is securely locked into the Micro Base before transport.

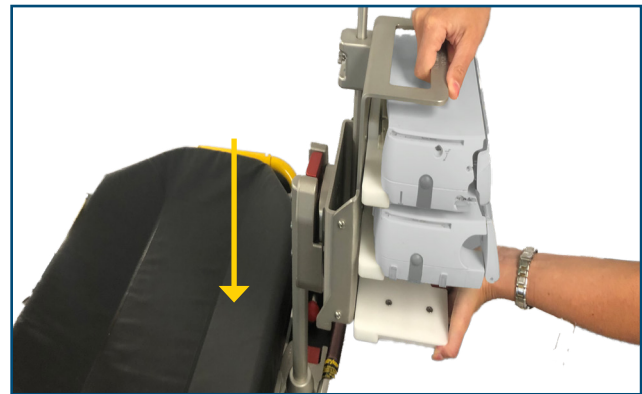

4. To remove the bracket from the Safety MD-Transporter, press the Red Quick Release Mechanism located on the Micro Base.
5. While holding the Red Quick Release Mechanism, lift the Technimount bracket upwards until removed and release the Red Quick Release Mechanism.



# MOUNTING SYSTEM PART NUMBERS

---

## KITS AND OTHER OPTIONS

DESCRIPTION	PART NUMBER
Part Number - Bracket Pro Serie 92-FL4 (Bracket for for Air Transport 4 Pumps)	2300-10-MTP-FL4
Part Number - Bracket Pro Serie 92-FL3 (Bracket for Air Transport 3 Pumps)	2300-10-MTP-FL3



### WARNING

Do not modify any components of these system. Modifying the products can cause unpredictable operation resulting in injury to the patient or operator.

### NOTE

Technimount reserves the right to change Part Numbers and Products without notice. Please refer to Customer Service to ensure product part number availability.

For any replacement parts or for repair, please contact Technical Support at [techsupport@technimount.com](mailto:techsupport@technimount.com)



# MAINTENANCE GUIDE

---

## CLEANING

### CLEANING PROCESS

---



#### WARNING

---

##### **ENSURE TO REMOVE MEDICAL DEVICE FROM THE BRACKET IF THOROUGH CLEANING IS NEEDED.**

Follow your EMS service's protocol or medical device manufacturer's user guide for the recommended cleaning procedure of your medical devices. Cleaning procedure below does NOT provide recommendations for cleaning of the medical device. The following cleaning guidelines consider the Technimount mounting assemblies and components.

It is the responsibility of each EMS service to use the cleaning solution of their choice. All EMS service's have policies and procedures describing how they should clean and disinfect their medical equipment. Use a chlorine bleach solution (3.25% - less than one part bleach to 100 parts water), a solution containing alcohol, phenolic cleaners (active ingredient - o-phenyl phenol) or a quaternary ammonium chloride solution. In diluted concentrations, these cleaning agents have no negative effect on our products.

The bracket can be cleaned by using a pressure washer. The bracket may show some signs of oxidation or discoloration from continuous washing, however, degradation of the product's performance or functionality will not be affected due to power washing if cleaning instructions are properly followed.

The material used to manufacture the bracket is high-grade aluminum, it is very durable and resistant to corrosion, however high levels of sodium or a high concentration of sodium or acid may prematurely corrode the product. Always rinse with soft soap and water if exposed.

- ✓ Use of a power washer can accelerate removal of contaminants collected during the use of the product.
- ✓ Rinse with clean water.
- ✓ Towel dry all components.
- ✓ Allow all other components to air dry.
- ✓ Avoid over saturation and ensure that the product does not stay wet longer than the cleaner manufacturer's guidelines for proper disinfecting.



#### WARNING

---

- ✓ When cleaning, always use appropriate personal protection equipment (PPE) based on established protocols (e.g., gloves, eye wear, etc.).



#### CAUTION

---

- ✓ Do not steam clean or use ultrasonic cleaners on the system or any of its components.
- ✓ Do not immerse the metal parts/components in water.
- ✓ Maximum water temperature should not exceed 180°F/82°C.
- ✓ Maximum water pressure should not exceed 500 psi/34,5 BAR. If a pressure washer is being used to clean the unit, the pressure nozzle must be kept a minimum of 24 inches (61 cm) from the unit.

# MAINTENANCE GUIDE

---

## CLEANING (CONTINUED)

### CLEANING SOLUTIONS

Phenolic type or quaternary type disinfectants can be used (excluding Virex® TB). Iodophor type disinfectants, Peracetic Acid, Chlorine, and especially Hydrogen Peroxide are not recommended for use because staining and corrosion may result.

Suggested cleaners for the bracket:

- ✓ Quaternary Cleaners (active ingredient - ammonium chloride)
- ✓ Water and soft soap
- ✓ A 10% diluted bleach solution, a solution containing alcohol or a quaternary ammonium chloride solution. In diluted concentrations, these cleaning agents have no negative effect on our products.



### WARNING

- ✓ Some cleaning products are corrosive in nature and may cause damage to the product if used improperly.
- ✓ If the products described are used to clean the equipment, the bracket and components must be rinsed with clean water and thoroughly dried following cleaning. Failure to properly rinse and dry the system will leave a corrosive residue on the surface of the product, possibly causing premature corrosion of critical components.

### REMOVAL OF IODINE COMPOUNDS

If rust appears on any surface of the product, we recommend sanding the affected surface to remove the rust and then using the cleaning method presented in this guide once the rust is removed. To clean the stained area, use a solution of 1/4 Tablespoon of Sodium Thiosulfate in 1 pint / .5 litre of warm water to clean the stained area. Clean as soon as possible after staining occurs. If stains are not immediately removed, allow solution to soak or stand on the surface. Thoroughly rinse surfaces with clean water and allow to air dry before returning unit to service.



### WARNING

- ✓ Failure to properly clean contaminated parts or components will increase the risk of exposure to blood borne pathogens and may cause injury to the patient or the operator.

## PREVENTIVE MAINTENANCE

A preventive and regular maintenance program should be established for all Technimount equipment. Preventive maintenance may be required more frequently based on the usage level of the product. The bracket requires regular preventative maintenance. Establish and follow a preventive maintenance schedule and keep records of preventive maintenance activities (see maintenance form in this user guide).



### CAUTION

- ✓ Close attention should be given to safety features including, but not limited to locking mechanisms and discs.
- ✓ Improper maintenance can cause injury or damage to the product. Maintain the product as described in this user guide.



# MAINTENANCE GUIDE

## INSPECTION PROCESS AND SCHEDULE

### MAINTENANCE PROGRAM

The following inspection routine and schedule is intended as a general guide for preventive maintenance of the Bracket. Factors such as weather, environment, geographical location, and individual usage will necessitate different maintenance. If you are unsure as to how to perform these maintenance inspections or at what interval to perform these inspections, please contact your Technical Support Team at [techsupport@technimount.com](mailto:techsupport@technimount.com). Check each routine and replace damaged or worn out parts if necessary.

Follow timetable below for maintenance (either in months (M) or hours (H)), in accordance with your Service's current maintenance practices and protocols.

<u>ITEM</u>	<u>ROUTINE</u>	<u>1 M</u> <u>OR</u> <u>2 H</u>	<u>3 M</u> <u>OR</u> <u>6 H</u>	<u>6 M</u> <u>OR</u> <u>12 H</u>	<u>12 M</u> <u>OR</u> <u>24 H</u>
Bracket	<ul style="list-style-type: none"><li>✓ Inspect Bracket<ul style="list-style-type: none"><li>▪ Inspect all screws</li><li>▪ Ensure it is working properly and there are no particles, obstacles, or debris</li><li>▪ Verify Discs</li><li>▪ Verify that there is no damage or dirt on the discs and there are no loose screws</li><li>▪ The discs can be easily inserted into and removed from Micro Base</li><li>▪ Verify the quick release mechanism of the pump holder to ensure it is working properly</li><li>▪ Lock and Unlock is working properly</li><li>▪ Verify IV pole, IV pole holder, and IV pole holder quick release knob are working properly</li></ul></li><li>✓ Do a visual inspection for the Bracket to ensure there is no damage or chemical attack</li></ul>	X			

[Return To Table of Contents](#)

[illegible]

# MAINTENANCE GUIDE

## TRAINING RECORD

<u>TRAINEE NAME</u>	<u>TRAINING DATE</u>		<u>TRAINING METHOD</u> <u>USER GUIDE,</u> <u>IN-SERVICE,</u> <u>IN-CLASS,</u> <u>ETC.</u>
	<u>BASIC TRAINING</u>	<u>TRAINING UPDATE</u>	

# MAINTENANCE GUIDE

---

## REPLACEMENT PARTS

The parts and accessories listed are all currently available for purchase. Some of the parts identified on the assembly drawing parts in this user guide may not be individually available for purchase. Please call Technimount Customer Service : +1.888.639.2758 or at [customerservice@technimount.com](mailto:customerservice@technimount.com) for availability and pricing.

<u>DESCRIPTION</u>	<u>PART NUMBER</u>
Part Number - Bracket Pro Serie 92-FL4 (Bracket for for Air Transport 4 Pumps)	2300-10-MTP-FL4
Part Number - Bracket Pro Serie 92-FL3 (Bracket for Air Transport 3 Pumps)	2300-10-MTP-FL3
IV Pole	TCM-052-051

# WARRANTY

---

## WARRANTY POLICY

This statement constitutes TECHNIMOUNT's entire warranty policy with regards to the TECHNIMOUNT Products.

TECHNIMOUNT MAKES NO OTHER WARRANTY OR REPRESENTATION NEITHER EXPRESSED NOR IMPLIED, EXCEPT AS SET FORTH HEREIN. THERE IS NO WARRANTY OF MERCHANTABILITY AND THERE ARE NO WARRANTIES OF FITNESS FOR ANY PARTICULAR PURPOSE. IN NO EVENT SHALL TECHNIMOUNT BE LIABLE HEREUNDER FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING FROM OR IN ANY MANNER RELATED TO SALES OR USE OF ANY SUCH EQUIPMENT.

TECHNIMOUNT E.M.S. Holding Inc. (TECHNIMOUNT) guarantees to the original "Purchaser" of the "Hardware" with which this "Limited Warranty" is included that the Hardware will be free from "Defects" in workmanship and materials under normal use for a "Warranty Period", one (1) year from the date that the Hardware was first purchased by the Purchaser. During the Warranty Period, the Hardware will be repaired or replaced according to the "Limited Warranty" without charge to the Purchaser for either parts or labor. The parts may be repaired or replaced with either new or refurbished parts. The product may be replaced for a new or refurbished product. For this Limited Warranty "refurbished" means parts and products, which have been returned to factory specifications. If the Hardware is repaired or replaced within the Warranty Period, the greater of the remaining warranty period will apply or three (3) months from the date of repair/replacement. If the Hardware is repaired or replaced after the Warranty Period has expired, the Warranty Period for the repair or replacement will expire three (3) months after the date of repair or replacement.

## LIMITED RESPONSIBILITY AND WARRANTY

The Limited Warranty does not apply to normal wear that should result from normal use. It does not apply when the Hardware or any component is opened, disassembled, or repaired by someone not authorized by TECHNIMOUNT and does not cover repair or replacement of any Hardware or part thereof damaged by neglect, misuse, moisture, liquids, exposure to heat, accidents, abuse, and non-compliance with the instructions for installation and use provided with the Hardware. The Limited Warranty does not cover physical damage to the surface of the Hardware, and the decision to repair, replace or refuse the coverage is final and its sole discretion of TECHNIMOUNT, and is made without any compensation or obligation from TECHNIMOUNT.

The hardware, or product defined as a mounting or bracket system for clipping and attaching medical equipment is specifically designed to fill this requirement. Any other use will void the warranty and TECHNIMOUNT will not be held liable on any claim if the product is modified or adapted for use.

Technimount products are intended to retain a medical device in place in the case of a single crash impact. A Technimount product must NOT be used again if it was involved in a crash. The product MUST be replaced. If the Purchaser uses a Technimount product following a crash, it is at the Purchaser's own risk and Technimount will not be held liable.

## INTERNATIONAL WARRANTY CLAUSE

This warranty reflects Canadian domestic policy. Warranty outside Canada may vary by country. Please contact TECHNIMOUNT Customer Service for more information.

# RETURN POLICY

---

## RETURN POLICY

Mounting systems and bracket systems for portable medical devices may be returned up to 60 days of receipt if:

- The product is not what was originally ordered
- The product does not meet specifications with TECHNIMOUNT's technical sheets according to the customer
- The product is not compatible with the equipment on which it was to be installed

For any manufacturing defect, refer to the conditions of the Warranty Policy.

### PRIOR TO 30 DAYS

- Product must be undamaged and in its original packaging
- The product return request must be provided in writing and it must be approved by TECHNIMOUNT prior to returning the product
- Returns will not be approved on a modified or damaged item
- Charges may apply if damaged or missing item
- Customer is responsible for a 10% restocking fee

### PRIOR 45 DAYS

- Product must be unused, undamaged and in its original packaging
- The product return request must be provided in writing and it must be approved by TECHNIMOUNT prior to returning the product
- Returns will not be approved on modified or damaged item
- Charges may apply if damaged or missing items
- Customer is responsible for a 25% restocking fee

### PRIOR 60 DAYS

- Product must be unused, undamaged and in its original packaging
- The product return request must be provided in writing and it must be approved by TECHNIMOUNT prior to returning the product
- Returns will not be approved on a modified or damaged item
- Charges may apply if damaged or missing items
- Customer is responsible for a 30% restocking fee

## RETURN AUTHORIZATION

TECHNIMOUNT's customer service department is responsible for approving any merchandise return and will provide a Return Merchandise Authorization (RMA) number to be printed on any returned merchandise. TECHNIMOUNT reserves the right to charge shipping and restocking fees on returned items. SPECIAL, MODIFIED, OR DISCONTINUED ITEMS NOT SUBJECT TO RETURN.

# RETURN POLICY

---

## DAMAGED MERCHANDISE

ICC Regulations require that claims for damaged merchandise must be made with the carrier within fifteen (15) days of receiving merchandise. DO NOT ACCEPT DAMAGED SHIPMENTS UNLESS SUCH DAMAGE IS NOTED ON THE DELIVERY RECEIPT AT THE TIME OF RECEIPT. Upon prompt notification, TECHNIMOUNT will file a freight claim with the appropriate carrier for damages incurred. Claim will be limited in amount to the actual replacement cost. In the event that this information is not received by TECHNIMOUNT within the fifteen (15) day period following the delivery of the merchandise, or the damage was not noted on the delivery receipt at the time of receipt, the customer will be responsible for payment of the original invoice in full.

Note: Claims for any short or broken shipment must be made within thirty (30) days of invoice.

## CLAIM PROCESS

Please follow the claim procedure below to return a defective product:

Fill in the RMA form online at <https://www.technimount.com/service-support/service-support-overview/>

### REQUIRED INFORMATION

- Serial number
- Invoice number
- Identification of the issue
- Upload the pictures of the issue
- Submit the form

A confirmation of the receipt of the claim will be sent to you within 2 business days. If you have not received a confirmation email by then, please contact [techsupport@technimount.com](mailto:techsupport@technimount.com).

If the RMA request is approved, TECHNIMOUNT will provide you with an RMA number for your claim. We will also include instructions for the replacement of your product. No product return will be accepted without the RMA number labeled on the outside of the box being returned.

### FINDINGS AND CONCLUSION

- If following an investigation, we find that our product is defective, TECHNIMOUNT will take corrective actions and close the claim
- If following an investigation, we find that our product is NOT defective (misuse or abuse of the product) the product will not be covered by the warranty. Details of our findings and conclusions will be provided to you

## QUESTIONS ABOUT OUR POLICY

If you have any questions regarding our return policy, please contact our Customer Service department at [customerservice@technimount.com](mailto:customerservice@technimount.com).

## SAE CERTIFICATION

---



---



# SAFETY AND FLEXIBILITY WHERE IT MATTERS MOST

TECHNIMOUNT E.M.S. HOLDING INC.  
3770 Jean Marchand Street, Suite 100-C  
Quebec (QC) G2C 1Y6  
Canada  
T + 1 581.700.6613  
TF + 1 888.639.2758  
[technimount.com](http://technimount.com)  
[info@technimount.com](mailto:info@technimount.com)

© 2022 Technimount E.M.S. Holding Inc. - All Rights Reserved  
Printed in Canada



**TECHNIMOUNT**  
EMS™

BP92-FL-UG-202206-R01